



## Measuring Guidelines for Facial Units

### Positioning Your Client When Measuring for Facial Garments

Tribute garments are most commonly worn while the client is sleeping, however, clients with facial swelling may be measured in a seated posture.

#### Measuring:

*Please record the needed measurements in the designated areas on the Facial Order Form.*

- A:** Measure the length from one ear tip to the other.
- B:** Measure the head circumference at the temple line or largest part.
- C:** Measure the distance from the outside crease of the right eye to the outside crease of the left eye.
- D:** Measure the length of the nose.
- E:** Measure the length at the midline from the nose to the upper lip.
- F:** Measure the length at the midline from the lower lip to the chin.
- G:** Measure the length from the sterno-clavicular notch to the chin. Follow the contour with chin in relaxed posture.
- H:** Measure from the right ear lobe to the right nostril.
- I:** Measure from the left ear lobe to the left nostril.
- J:** Measure the distance from the right TMJ to the chin.
- K:** Measure the distance from the left TMJ to the chin.
- L:** Measure the length from the right ear lobe to the HPS (High Point of Shoulder).
- M:** Measure the length from the left ear lobe to the HPS.
- N:** Measure the neck circumference.