

HOW TO MEASURE FOR

TributeNight™ Head & Neck Garments

①

Mark the length reference points from where you would like the garment to start and end at the determined positions directly on your patient's skin using a water-based marker.

②

Measure the circumferences in centimeters at those same marked reference points and record the measurements on the left side of the diagram on your Order Form. *(See details below.)*

③

Measure the length at each marked reference point, and record the measurements on the right side of the diagram on your Order Form. *(See details below.)*

We recommend patients be in a relaxed, supine (laying down) position.

Answers to Frequently Asked Questions

- ⌘ In extreme presentations, measure the lobules, breast and/or pannus in a supported position.
- ⌘ When skin folds are present, measure the smallest and largest circumferences.
- ⌘ Contact our Custom Design Center to send photos or illustrations to further explain patient presentations.
- ⌘ With patients who fluctuate frequently, please feel free to add a circumference range.

Circumferences



Measure **B** circumference around the forehead.



Measure **N** circumference of the neck at its greatest circumference.

Lengths



Measure **A** length over the top of the head from one ear tip to the other ear tip.



Measure the horizontal **C** length from the outer crease of the right eye to the outer crease of the left eye.



Measure **D** length of the nose from the bridge to the tip.



Measure **E** length from the base of the nose to the top of the upper lip.



Measure **F** length from the bottom of the lower lip to the tip of the chin.



Measure **G** length from the tip of the chin to the sternoclavicular notch, following the contour of the neck with the chin in relaxed position.



Measure the horizontal **H** length from the right ear lobe crease to the right nostril crease.



Measure the horizontal **I** length from the left ear lobe crease to the left nostril crease.



Measure **J** length from the right TMJ to the midpoint of the chin, following the mandibular contour.



Measure **K** length from the left TMJ to the midpoint of the chin, following the mandibular contour.



Measure **L** length from the right ear lobe to the high point of shoulder (HPS).



Measure **M** length from the left ear lobe to the high point of shoulder (HPS).

Questions?

Please contact the Custom Design Center, available Monday–Friday, 7:00 AM–7:00 PM Central

☎ 414-892-5158

✉ customdesigncenter@us.LRmed.com