

HOW TO MEASURE FOR

TributeNight™ Hand Garments

①

Trace the hand on the order form, or a separate piece of paper.

②

Measure the length at each marked reference point, and record the measurements on the diagram on your Order Form. *(See details below.)*

We recommend patients be in a relaxed, supine (laying down) position.

Answers to Frequently Asked Questions

When skin folds are present, measure the smallest and largest circumferences.

Contact our Custom Design Center to send photos or illustrations to further explain patient presentations.

With patients who fluctuate frequently, please feel free to add a circumference range.

Circumferences

A_c Optional: In a relaxed position, measure the distal circumference at the determined **A** position (digits #2–5).

B_c Measure the circumference at the determined **B** position (MCP) on the palm of the hand.

C_c Measure the circumference at the determined **C** position (wrist over the styloid).

D_c Measure the proximal circumference at the determined **D** position (#1 digit/thumb).

E_c Measure the proximal circumference at the determined **E** position (#2 digit/index finger).

F_c Measure the proximal circumference at the determined **F** position (#3 digit/middle finger).

G_c Measure the proximal circumference at the determined **G** position (#4 digit/ring finger).

H_c Measure the proximal circumference at the determined **H** position (#5 digit/small finger).