

HOW TO MEASURE FOR

TributeNight™ Hand Garments

①

Trace the hand on the order form, or a separate piece of paper.

②

Measure the length at each marked reference point, and record the measurements on the diagram on your Order Form. *(See details below.)*

We recommend patients be in a relaxed, supine (laying down) position.

Answers to Frequently Asked Questions

- ⌘ When skin folds are present, measure the smallest and largest circumferences.
- ⌘ Contact our Custom Design Center to send photos or illustrations to further explain patient presentations.
- ⌘ With patients who fluctuate frequently, please feel free to add a circumference range.

Circumferences

A_c Optional: In a relaxed position, measure the distal circumference at the determined **A** position (digits #2–5).

B_c Measure the circumference at the determined **B** position (MCP) on the palm of the hand.

C_c Measure the circumference at the determined **C** position (wrist over the styloid).

D_c Measure the proximal circumference at the determined **D** position (#1 digit/thumb).

E_c Measure the proximal circumference at the determined **E** position (#2 digit/index finger).

F_c Measure the proximal circumference at the determined **F** position (#3 digit/middle finger).

G_c Measure the proximal circumference at the determined **G** position (#4 digit/ring finger).

H_c Measure the proximal circumference at the determined **H** position (#5 digit/small finger).