



HOW TO MEASURE FOR

TributeNight™ Head & Neck Garments

①

Mark the length reference points from where you would like the garment to start and end at the determined positions directly on your patient's skin using a water-based marker.

②

Measure the circumferences in centimeters at those same marked reference points and record the measurements on the left side of the diagram on your Order Form. *(See details below.)*

③

Measure the length at each marked reference point, and record the measurements on the right side of the diagram on your Order Form. *(See details below.)*

We recommend patients be in a relaxed, supine (laying down) position.

Answers to Frequently Asked Questions

In extreme presentations, measure the lobules, breast and/or pannus in a supported position.

When skin folds are present, measure the smallest and largest circumferences.

Contact our Custom Design Center to send photos or illustrations to further explain patient presentations.

With patients who fluctuate frequently, please feel free to add a circumference range.

Circumferences



Measure **B** circumference around the forehead.



Measure **N** circumference of the neck at its greatest circumference.

Lengths



Measure **A** length over the top of the head from one ear tip to the other ear tip.



Measure the horizontal **C** length from the outer crease of the right eye to the outer crease of the left eye.



Measure **D** length of the nose from the bridge to the tip.



Measure **E** length from the base of the nose to the top of the upper lip.



Measure **F** length from the bottom of the lower lip to the tip of the chin.



Measure **G** length from the tip of the chin to the sternoclavicular notch, following the contour of the neck with the chin in relaxed position.



Measure the horizontal **H** length from the right ear lobe crease to the right nostril crease.



Measure the horizontal **I** length from the left ear lobe crease to the left nostril crease.



Measure **J** length from the right TMJ to the midpoint of the chin, following the mandibular contour.



Measure **K** length from the left TMJ to the midpoint of the chin, following the mandibular contour.



Measure **L** length from the right ear lobe to the high point of shoulder (HPS).



Measure **M** length from the left ear lobe to the high point of shoulder (HPS).

Questions?

Please contact the Custom Design Center, available Monday–Friday, 7:00 AM–7:00 PM Central

☎ 414-892-5158

✉ customdesigncenter@us.LRmed.com