

HOW TO MEASURE FOR

TributeNight™ Torso Garments

①

Mark the length reference points from where you would like the garment to start and end at the determined positions directly on your patient's skin using a water-based marker.

②

Measure the circumferences in centimeters at those same marked reference points and record the measurements on the left side of the diagram on your Order Form. *(See details below.)*

③

Measure the length at each marked reference point, and record the measurements on the right side of the diagram on your Order Form. *(See details below.)*

We recommend patients be in a relaxed, supine (laying down) position.

Answers to Frequently Asked Questions

In extreme presentations, measure the lobules, breast and/or pannus in a supported position.

When skin folds are present, measure the smallest and largest circumferences.

Contact our Custom Design Center to send photos or illustrations to further explain patient presentations.

With patients who fluctuate frequently, please feel free to add a circumference range.

Circumferences

Left side of diagram

- H_c** Measure hip circumference at determined **H** position (Iliac Crest).
- I_c** Measure waist circumference at determined **I** (natural waistline/zero position)
- J_c** Measure circumference at determined **J** position (just below chest). Lift chest if necessary.
- K_c** Measure chest circumference at determined **K** position (mid-breast/fullest part of breast line). For unilateral mastectomy, measure left and right side independently from anterior midline of chest to posterior midline.
- L_c** Measure circumference at determined **L** position (superior chest).
- N_c** Measure **N** circumference of the neck at its greatest circumference.

Lengths

Right side of diagram

- H^L** Measure length from **I** (natural waistline/zero position) below transverse watershed to **H** position (Iliac Crest).
- I^L** Zero (0) position. No measurement needed.
- J^L** Measure length from **I** (natural waistline/zero position) to **J** (just below breast).
- K^L** Measure length from **I** (natural waistline/zero position) to **K** (mid-breast/fullest part of the breast line).
- L^L** Measure length from **I** (natural waistline/zero position) to **L** (axilla). *A pencil can be placed in the armpit as a visual guide.*
- M^L** Measure shoulder length of **M** from HPS (high point shoulder/base of neck) to AC Joint.
- N^L** Measure shoulder straddle length of **N** from determined anterior **I** (natural waistline/zero position), over shoulder and back to posterior **I** (natural waistline/zero position)
- O^L** Measure length of **O** from AC joint, under axilla and back to starting AC joint position.