

PURE NIGHT

COMPRESSION THERAPY REIMAGINED

PURE MEDICAL
OUR COMMITMENT. YOUR COMFORT.


OUR COMMITMENT
TO YOUR COMFORT IS
KNITTED INTO EVERY
GARMENT WE MAKE.



PURE NIGHT CUSTOM COMPRESSION GARMENT

With patented flat-knit technology, Pure Medical's Pure Night garment brings control to the medical compression garment industry by producing a mechanically-structured nighttime garment tailored to each patient's gradient pressure needs.

Unlike foam garments, Pure Night's flat-knit technology distributes gradient pressure with breathable microfiber filling, resulting in a garment that's easy to use, breathable, and comfortable for patients looking to reduce fluid build-up and manage lymphedema.

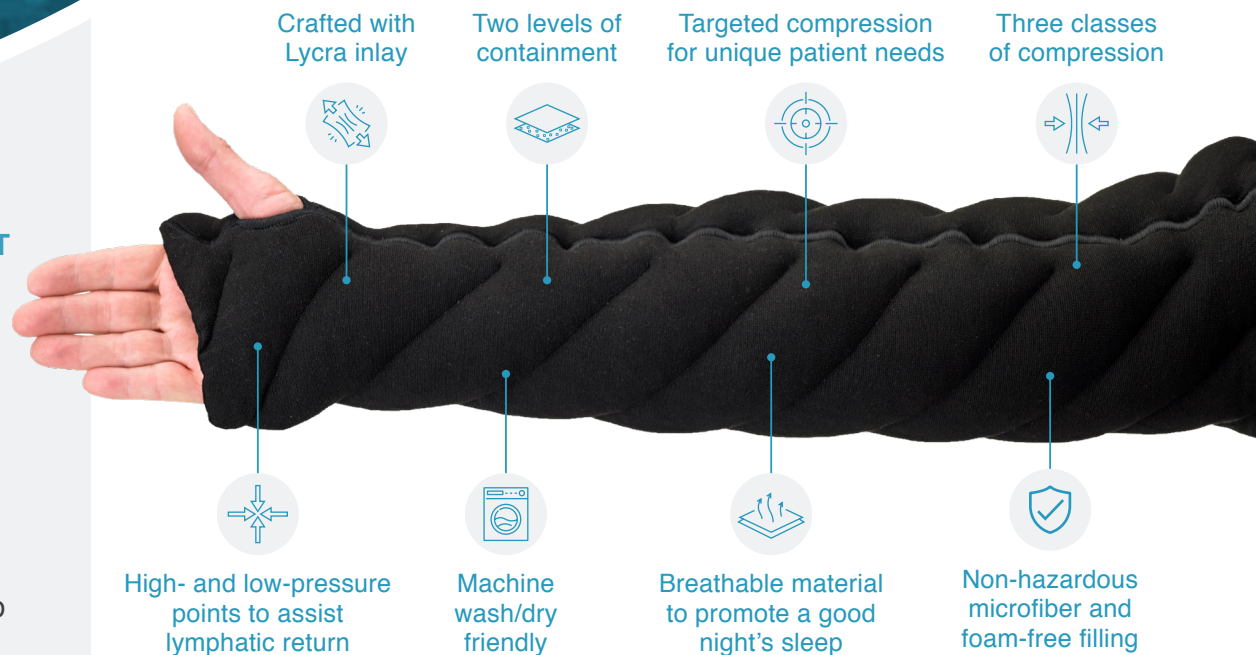



WHAT IS CONTAINMENT?

Containment is the stiffness factor of the garment. The stiffer the garment, the more containment it provides. Containment assists the muscle pump action that naturally occurs when muscles flex and relax against the skin.

WHAT IS COMPRESSION?

Compression is the amount of pressure the garment will apply in a rest position. Gradient compression is critical in improving the lymphatic flow.

HOW DO THESE TWO FACTORS WORK TOGETHER?

Higher levels of containment result in higher amounts of pressure. Too much stiffness can amount to ineffectively-applied pressure and difficulty applying the garment for the patient. We're here to help therapists and patients alike find the right balance of containment and compression to effectively treat each unique condition.

CAUTION SHOULD BE USED WITH THE FOLLOWING CONDITIONS:

- Arterial insufficiency
- Deep vein thrombosis (DVT)
- Congestive heart failure or heart disease
- Infection
- Impaired sensation of the affected extremity
- Cognitive impairment

COMPRESSION	CONTAINMENT			
	CHRONIC SWELLING		SPRAINS	
	1	2	1	2
20–30 mmHg	Prophylactic	Mild Indurated	-	Grade 1
30-40 mmHg	Mild	Moderate Indurated	-	Grade 2
40-50 mmHg	Moderate	Aggressive Indurated	-	Grade 3

Consult with a medical professional to determine what levels are appropriate for you.

For more information, contact Pure Medical customer service at info@puremedical.us or via cell **414-732-6515**.

Visit puremedical.us or scan QR code.

