

SIGVARIS
GROUP

Compression Solutions



Graduated Compression
Socks & Hosiery for
Veno-Lymphatic Disorders

Why choose Sigvaris compression solutions?

1. Easy to put on

Our hosiery uses double-covered LYCRA® spandex inlay yarns so garments glide on easily.

2. Great fit

Our sizing system has a wide range of sizes for an exceptional fit.

3. Easy to care for

Machine wash and dry makes laundering easy.

4. Quality

We guarantee the compression of our products for six months.

5. Lots of choices

We offer you more choices of materials, colors, and patterns than any other medical compression brand.



Live your life.
Be yourself.

SIGVARIS
GROUP

How to Order Medical Products

Series Number 23 = Cotton (See individual Catalog pages for series number)	Compression Level 1 = 15–20mmHg 2 = 20–30mmHg* 3 = 30–40mmHg 4 = 40–50mmHg** 5 = 50–60mmHg**	Product Style C = Knee-high or Calf (See individual Catalog pages for available styles)	Size ML = Medium Long (See individual Catalog pages for sizing)	Gender M = Men W = Women O = Open toe	Color 99 = Black (See individual Catalog pages for available colors)
23	2	C	ML	M	99

* 18–25mmHg available in Diabetic Compression Socks only

** 40–50mmHg available in Secure stocking; 40–50mmHg and 50–60mmHg available in Natural Rubber only

To order Plus Calf with Grip-top or Plus Size Pantyhose:
add “/P” after the color in the ordering code (860P for women only)

To order Calf Style with Grip-top:
add “/S” after the color in the ordering code (see product Catalog pages for availability)

To order Thigh with Waist Attachment:
add “/L” to indicate the Left leg or “/R” to indicate the Right leg after the color in the ordering code (860W and 500W only)

To order Motion High Tech and Motion Thermoregulating Wool:
the letter for Gender is not used in the ordering code.

Example: 412CSM59 = Motion High Tech, 20–30mmHg, Closed toe Calf, Size Small Medium, Color Red

How to Order WELL BEING Products

Series Number 120 = SHEER FASHION (See individual Catalog pages for series number)	Product Style N = Thigh-high (See individual Catalog pages for available styles)	Size A = Small (See individual Catalog pages for sizing)	Color 33 = Honey (See individual Catalog pages for available colors)
120	N	A	33

To order ATHLETIC RECOVERY SOCKS, PERFORMANCE SLEEVES, MERINO OUTDOOR SOCKS, and EVERSOFIT DIABETIC SOCKS:

Size is only one letter and the letter for Gender is not used in the ordering code.

Example: 421CL31 = MERINO OUTDOOR SOCKS, 15–20mmHg, Closed toe Calf, Size Large, Color Olive

To Order:
1.800.322.7744, ext. 1

Online Dealer Portal:
sigvaris-online.com

Please follow these instructions carefully to determine the correct size for different product styles.

Measurements should always be taken in the following order, and as soon after waking as possible when swelling is minimal.

CALF STYLE

Open/Closed Toe

1	Measure the ankle circumference at the narrowest part of the ankle, above the ankle bone. Use the appropriate series sizing chart to determine if ankle is SMALL, MEDIUM, LARGE or X-LARGE. Note: This should be your first measurement.
2	Measure the calf circumference at the fullest part of the calf.
3	Determine the calf length by measuring from the floor up to the fibular head. This measurement is taken on the outside of the leg.

PANTYHOSE AND THIGH-HIGH STYLE

Open/Closed Toe

1	Measure the ankle circumference directly above the ankle bone. Use the appropriate series sizing chart to determine if ankle is SMALL, MEDIUM, LARGE, or X-LARGE. Note: This should be your first measurement.
2	Measure the calf circumference at the fullest part of the calf.
3	Skip Step 3 (but refer to General Guidelines #6 below).
4	Measure the thigh at its widest circumference.
5	Determine the leg length by measuring from the floor up to the gluteal fold.
6	Determine the hip circumference by measuring the widest part of the hips.

GENERAL GUIDELINES

1	Measurements should always be taken as soon after waking as possible, when swelling is minimal.
2	Measure against bare skin, not over clothing.
3	The measuring system is based on having the foot flat against the floor and the calf or leg at a right angle to the foot (see images to the right for examples).
4	Measurements should be taken by pulling the tape around the leg so that it is snug but not constricting.
5	Length measurements should be taken from the ground up.
6	Take all measurements, even if the patient only requires calf-length products, as the patient might decide at a later date to purchase additional styles.




1 Ankle circumference directly above ankle bone



2 Calf circumference at fullest part of the calf



3 Calf length from the floor to the fibular head

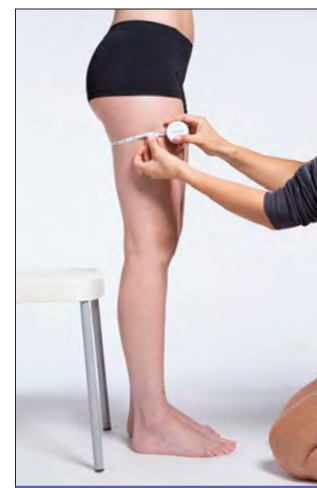

4 Widest circumference of the thigh



5 Leg length from the floor up to the gluteal fold



6 Circumference at the hip



SIGVARIS
GROUP

Established in 1864, SIGVARIS GROUP is, and has always been, a family-owned company. Founded and headquartered in Switzerland, our passion for helping people feel their best is why we've been a global market leader for 158 years and counting.

Our products restore, assist, and enhance people's lives through brands like:

sigvaris

Medical Compression Socks, Hosiery,
and Solutions for Medically Complex Edema

SIGVARIS WELL BEING

Consumer Healthcare: Simplified Sizing
Compression Socks and Hosiery

We are renowned for making exceptional products that combine advanced technology and quality craftsmanship with diverse styles that fit our modern customers' lives.

When heritage, competence, and dedication come together, there's no limit to where we'll go.

Live your life. Be yourself.

For more information, please visit:
sigvaris.com

	SERIES	PAGE	COMPRESSION LEVELS	STYLES
Essential	Cotton (230)	6	20–30mmHg 30–40mmHg	closed toe: calf, calf w/grip-top, thigh w/grip-top open toe: calf, thigh w/grip-top
	Opaque (860) <i>(for women)</i>	8	20–30mmHg 30–40mmHg	closed toe: calf, petite calf, calf plus, calf w/grip-top, thigh w/grip-top, petite thigh w/grip-top, extra-large thigh w/grip-top, pantyhose, petite pantyhose, plus-sized pantyhose open toe: calf, calf plus, calf w/grip-top, thigh w/grip-top, extra-large thigh w/grip-top, thigh w/waist attachment
	Opaque (860) <i>(for men)</i>	10	20–30mmHg 30–40mmHg	closed toe: calf, calf plus, calf w/grip-top, thigh w/grip-top, extra-large thigh w/grip-top, open toe: calf, calf plus, calf w/grip-top, thigh w/grip-top, extra-large thigh w/grip-top, thigh w/waist attachment
Style	Sea Island Cotton (220)	14	20–30mmHg	closed toe: calf
	Patterns (710) <i>(for women only)</i>	16	20–30mmHg	closed toe: thigh w/grip-top, pantyhose
	Medium Sheer (750) <i>(for women only)</i>	18	20–30mmHg	closed toe: calf, thigh w/grip-top, pantyhose
	Sheer (780) <i>(for women only)</i>	20	15–20mmHg 20–30mmHg 30–40mmHg	closed & open toe: calf, thigh w/grip-top, pantyhose, maternity pantyhose
	Microfiber (820) <i>(for men only)</i>	22	15–20mmHg 20–30mmHg 30–40mmHg	closed toe: calf, calf w/grip-top, thigh w/grip-top, extended foot sizes
	Microfiber Patterns (830)	24	20–30mmHg	closed toe: calf
	Soft Opaque (840) <i>(for women only)</i>	26	15–20mmHg 20–30mmHg 30–40mmHg	closed toe: calf, calf w/grip-top, thigh w/grip-top, pantyhose, maternity pantyhose open toe: calf, thigh w/grip-top, pantyhose
Motion	Cushioned Cotton (360)	30	20–30mmHg	closed toe: calf
	High Tech (412)	32	20–30mmHg	closed toe: calf
	Thermoregulating Wool (422)	34	20–30mmHg	closed toe: calf
Specialty	Natural Rubber (500)	38	30–40mmHg 40–50mmHg 50–60mmHg	open toe: calf, thigh, thigh w/grip-top, thigh w/waist attachment, pantyhose
	Secure (550)	40	20–30mmHg 30–40mmHg 40–50mmHg	closed & open toe: calf w/grip-top, thigh w/grip-top
	Secure Armsleeve, Glove, and Gauntlet (560)	42	15–20mmHg 20–30mmHg 30–40mmHg	armsleeve w/ grip-top (glove and gauntlet available separately)
	Diabetic Compression Socks (602)	44	18–25mmHg	closed toe: calf
	Advance Armsleeve (910)	46	15–20mmHg 20–30mmHg 30–40mmHg	armsleeve w/ grip-top w/ gauntlet, armsleeve w/ grip-top w/out gauntlet

SERIES	PAGE	COMPRESSION LEVEL	STYLES
CASUAL COTTON (146/186)	50	15–20mmHg	closed toe: calf
CUSHIONED COTTON (142/182)	52	15–20mmHg	closed toe: calf
SEA ISLAND COTTON (151/191)	54	15–20mmHg	closed toe: calf
ALL-SEASON MERINO WOOL (152/192)	56	15–20mmHg	closed toe: calf
MICROFIBER SHADES (143/183)	58	15–20mmHg	closed toe: calf
SHEER FASHION (120) <i>(for women only)</i>	60	15–20mmHg	closed toe: calf, thigh w/grip-top, panty hose open toe: calf
SOFT SILHOUETTE LEGGINGS (170) <i>(for women only)</i>	62	15–20mmHg	open foot: leggings, maternity leggings
ATHLETIC RECOVERY SOCKS (401)	64	15–20mmHg	closed toe: calf
PERFORMANCE SLEEVES (412)	66	20–30mmHg	open foot: calf sleeve
MERINO OUTDOOR SOCKS (421)	68	15–20mmHg	closed toe: calf
EVERSOFT DIABETIC SOCKS (160)	70	8–15mmHg	closed toe: calf

Medically Complex Edema	74
Dynaven	84
Accessories	90

Custom sizes are available for a cotton product, a microfiber product, and our Natural Rubber series.

For more information, please speak with your
SIGVARIS GROUP Sales Representative or
Customer Care: **1.800.322.7744**, ext. 1.

sigvaris



ESSENTIAL
Support Yourself

- Why choose **Essential**?
- Everyday comfort and reliability
 - Classic look and feel
 - Our widest range of sizes

Essential Products		
Product	Series Number	Page
Cotton	230	6
Opaque for women	860	8
Opaque for men	860	10

230

Essential COTTON

for women & men

Essential Cotton combines all-day cotton comfort and a classic ribbed design that is ideal for sensitive skin.

Compression Level

20-30mmHg

30-40mmHg

Features and Benefits



**Sensitive
Skin**



Cotton



Breathable

- Supima® cotton for highest level of comfort and breathability
- Ideal for sensitive skin and allergies
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off

Fiber Content

- 50% Nylon, 25% Spandex, 25% Supima Cotton (content can vary slightly by style)
- Non-latex

SUPIMA®
WORLD'S FINEST COTTONS

Best Suited for

- Containment of various types of edema, including venous edema, phlebolympheidema, and early stages of lymphedema
- Ideal for dress or casual shoes
- Ideal for long-distance travel



Silicone
band grip-top
in calf and thigh

20-30mmHg
Closed Toe Calf w/ Grip-top
Medium Long in Black

232CMLM99/S



20-30mmHg
Closed Toe Thigh Small Long in Light Beige

232NSLW66

Sizing Chart

	Small		Medium		Large		X Large	
Ankle Circumference	7"-8.5" (18-21.5cm)		8.5"-10" (21.5-25.5cm)		10"-11.5" (25.5-29cm)		11.5"-14" (29-36cm)	
Calf Circumference	11"-15.5" (28-39cm)		12.5"-17.5" (31.5-45cm)		14"-20" (35.5-51cm)		16"-24" (41-61cm)	
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	18"-26" (45.5-66cm)		19"-28.5" (48-73cm)		20.5"-33" (52-84cm)		NA	
Leg Length: Women & Open Toe	Short	Long	Short	Long	Short	Long		
	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)		
Leg Length: Men	Short	Long	Short	Long	Short	Long		
	≤30" (76cm)	>30" (76cm)	≤30" (76cm)	>30" (76cm)	≤30" (76cm)	>30" (76cm)		
Size	SS	SL	MS	ML	LS	LL	XS	XL

Women's Shoe Sizes: Short Leg 5.5-9.5, Long Leg 8-12 Men's Shoe Sizes: Short Leg 7-11, Long Leg 9.5-14

Styles and Colors

<div>Closed Toe = ●</div> <div>Open Toe = ○</div>				
<div>WHITE 00</div> <div>NAVY 10</div> <div>LIGHT BEIGE 66</div> <div>BLACK 99</div>				
Calf (C)	○	●	●○	●
Calf w/Grip-Top (C/S) <small>Silicone band grip-top</small>			●*	●**
Thigh w/Grip-Top (N) <small>Silicone band grip-top</small>			●○	●

* 20-30mmHg only
** 30-40mmHg calf with grip-top only available in black, 30-40mmHg extra large calf only available in black with grip-top

Product Code Example: 232CMLM10

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color
23	2	C	ML	M	10

Product Code Example: 232CLSM66/S

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color	Grip-top
23	2	C	LS	M	66	/S

Essential

OPAQUE

for women

Essential Opaque offers durability you can trust in our largest range of sizes, making it your go-to choice for everyday hosiery.

(For Men's Essential Opaque, see page 10)

Compression Level

20–30mmHg

30–40mmHg

Features and Benefits


Durable

**Largest
Size Range**

Opaque

- Double-covered inlay yarns provide comfort and allow product to easily glide on and off
- Cotton-lined gusset for pantyhose styles
- Pantyhose plus style is a great option during pregnancy

Fiber Content

- 64% Nylon, 36% Spandex (content can vary slightly by style)
- Non-latex

Best Suited for


- Containment of various types of edema including venous edema, phlebolympheidema, and early stages of lymphedema
- Post-vein treatment including thermal and non-thermal techniques
- Patients looking for petite and plus-sized options



Silicone band
grip-top
in thigh and
calf w/ Grip-top

20–30mmHg
Closed Toe Thigh Medium Long in Golden

862NMLW36

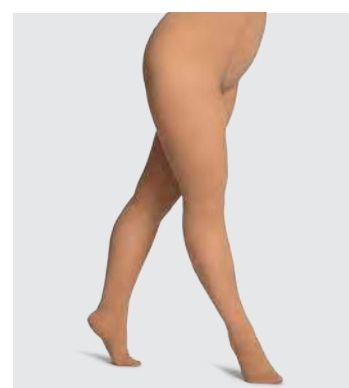

30–40mmHg
Closed Toe Pantyhose Small Short in Honey

863PMSW33

Women's Sizing Chart (for Men's Sizing Chart, see page 11)

	Small			Medium			Large		X Large	
Ankle Circumference	7"-8.5" (18-21.5cm)			8.5"-10" (21.5-25.5cm)			10"-11.5" (25.5-29cm)		11.5"-14" (29-36cm)	
Calf Circumference	11"-15.5" (28-39cm)			12.5"-17.5" (31.5-45cm)			14"-20" (35.5-51cm)		16"-24" (41-61cm)	
Calf Circumference: Calf Plus w/Grip-Top	NA			15.5"-21" (39.5-53cm)			18"-24" (45.5-61cm)		20"-28" (51-71cm)	
Calf Length	Petite	Short	Long	Petite	Short	Long	Short	Long	Short	Long
	≤14" (35.5cm)	≤16" (41cm)	>16" (41cm)	≤14" (35.5cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	18"-26" (45.5-66cm)			19"-28.5" (48-73cm)			20.5"-33" (52-84cm)		25"-37.5" (64-95cm)	
Leg Length: Women & Open Toe	Petite	Short	Long	Petite	Short	Long	Short	Long	Short	Long
	≤25.5" (65cm)	≤28.5" (72cm)	>28.5" (72cm)	≤25.5" (65cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)
Hip Circumference Guidelines	32.5"-44.5" (83-112.5cm)			34.5"-50" (87-127cm)			38.5"-59.5" (98-150.5cm)		NA	
Size	SP	SS	SL	MP	MS	ML	LS	LL	XS	XL
Calf Plus w/Grip-Top					MS/P	ML/P	LS/P	LL/P	XS/P	XL/P

Women's Shoe Sizes: Petite Leg 4-8,
Short Leg 5.5-9.5, Long Leg 8-12

Plus (Maternity) Hip Circumference Guidelines
Small ≤56" (142cm), Medium ≤62" (157.5cm), Large ≤70" (178cm)

Women's Styles and Colors (for Men's Styles and Colors, see page 11)

Closed Toe = ●

Open Toe = ○ (open toe products are opaque hosiery suitable for women or men)

HONEY 33	GOLDEN 36	LIGHT BEIGE 66	BLACK 99
-------------	--------------	----------------------	-------------

Women's Calf (C)	●	●	○	●●
Women's Calf Petite Sizes (C) Only available in 20-30mmHg, not available with silicone band grip-top	●	●		●
Women's Calf w/Grip-Top (C/S) Silicone band grip-top			●○	●
Women's Calf Plus w/Grip-Top (C/P) Only available with silicone band grip-top			●○	
Women's Thigh w/Grip-Top (N) Silicone band grip-top*	●	●	○	●
Women's Pantyhose (P)	●	●		●
Women's Pantyhose Plus (P/P)			●	●
Thigh w/Waist Attachment (W)** Indicate leg LEFT (/L) or RIGHT (/R)			○	

* Women's Thigh w/Grip-Top (N) and Open Toe Thigh w/Grip-Top (N) sizes XS and XL are only available in Light Beige

** Thigh w/Waist Attachment not available in XL sizes

Product Code Example: 862CSLW99

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color
86	2	C	SL	W	99

Product Code Example: 862CLLW66/P

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color	Grip-top
86	2	C	LL	W	66	/P

Essential

OPAQUE

for men

Essential Opaque offers durability you can trust in our largest range of sizes, making it your go-to choice for everyday hosiery.

(For Women's Essential Opaque, see page 8)

Compression Level

20-30mmHg

30-40mmHg

Features and Benefits



Durable



Largest Size Range



Opaque

- Double-covered inlay yarns provide comfort and allow product to easily glide on and off
- Men's calf products feature ribbed fabric; open toe products are a smooth hosiery fabric
- Flat, low friction toe seam

Fiber Content

- 64% Nylon, 36% Spandex (content can vary slightly by style)
- Non-latex

Best Suited for

- Containment of various types of edema including venous edema, phlebolympheidema, and early stages of lymphedema
- Post-vein treatment including thermal and non-thermal techniques
- Patients looking for plus-sized options



Ribbed fabric on Men's closed toe Essential Opaque products



Silicone band grip-top in thigh and calf w/ Grip-top

20-30mmHg
Open Toe Thigh Medium Long in Light Beige

862NML066



20-30mmHg
Open Toe Calf Medium Long in Black

862CML099


Men's Sizing Chart (for Women's Sizing Chart, see page 9)

	Small		Medium		Large		X Large	
Ankle Circumference	7"-8.5" (18-21.5cm)		8.5"-10" (21.5-25.5cm)		10"-11.5" (25.5-29cm)		11.5"-14" (29-36cm)	
Calf Circumference	11"-15.5" (28-39cm)		12.5"-17.5" (31.5-45cm)		14"-20" (35.5-51cm)		16"-24" (41-61cm)	
Calf Circumference: Calf Plus w/Grip-Top	NA		15.5"-21" (39.5-53cm)		18"-24" (45.5-61cm)		20"-28" (51-71cm)	
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	18"-26" (45.5-66cm)		19"-28.5" (48-73cm)		20.5"-33" (52-84cm)		25"-37.5" (64-95cm)	
Leg Length: Men	Short	Long	Short	Long	Short	Long	Short	Long
	≤30" (76cm)	>30" (76cm)	≤30" (76cm)	>30" (76cm)	≤30" (76cm)	>30" (76cm)	≤30" (76cm)	>30" (76cm)
Leg Length: Open Toe	Short	Long	Short	Long	Short	Long	Short	Long
	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)
Size	SS	SL	MS	ML	LS	LL	XS	XL
Calf Plus w/Grip-Top			MS/P	ML/P	LS/P	LL/P	XS/P	XL/P

Men's Shoe Sizes: Short Leg 7-11, Long Leg 9.5-14

Men's Styles and Colors (for Women's Styles and Colors, see page 9)

Closed Toe = ● (closed toe men's products are ribbed hosiery)
Open Toe = ○ (open toe products are opaque hosiery suitable for women or men)

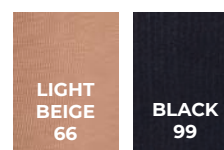

Men's Calf (C)	●○	●●
Men's Calf w/Grip-Top (C/S) Silicone band grip-top	●○	●
Men's Calf Plus w/Grip-Top (C/P) Only available with silicone band grip-top	●○	●
Men's Thigh w/Grip-Top (N) Silicone band grip-top	●○	●
Thigh w/Waist Attachment (W)** Indicate leg LEFT (/L) or RIGHT (/R)	○	


* Open Toe Thigh w/Grip-Top (N) sizes XS and XL are only available in Light Beige

** Thigh w/Waist Attachment not available in XL sizes

Product Code Example: 863CMS99

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color
86	3	C	MS	M	99

sigvaris

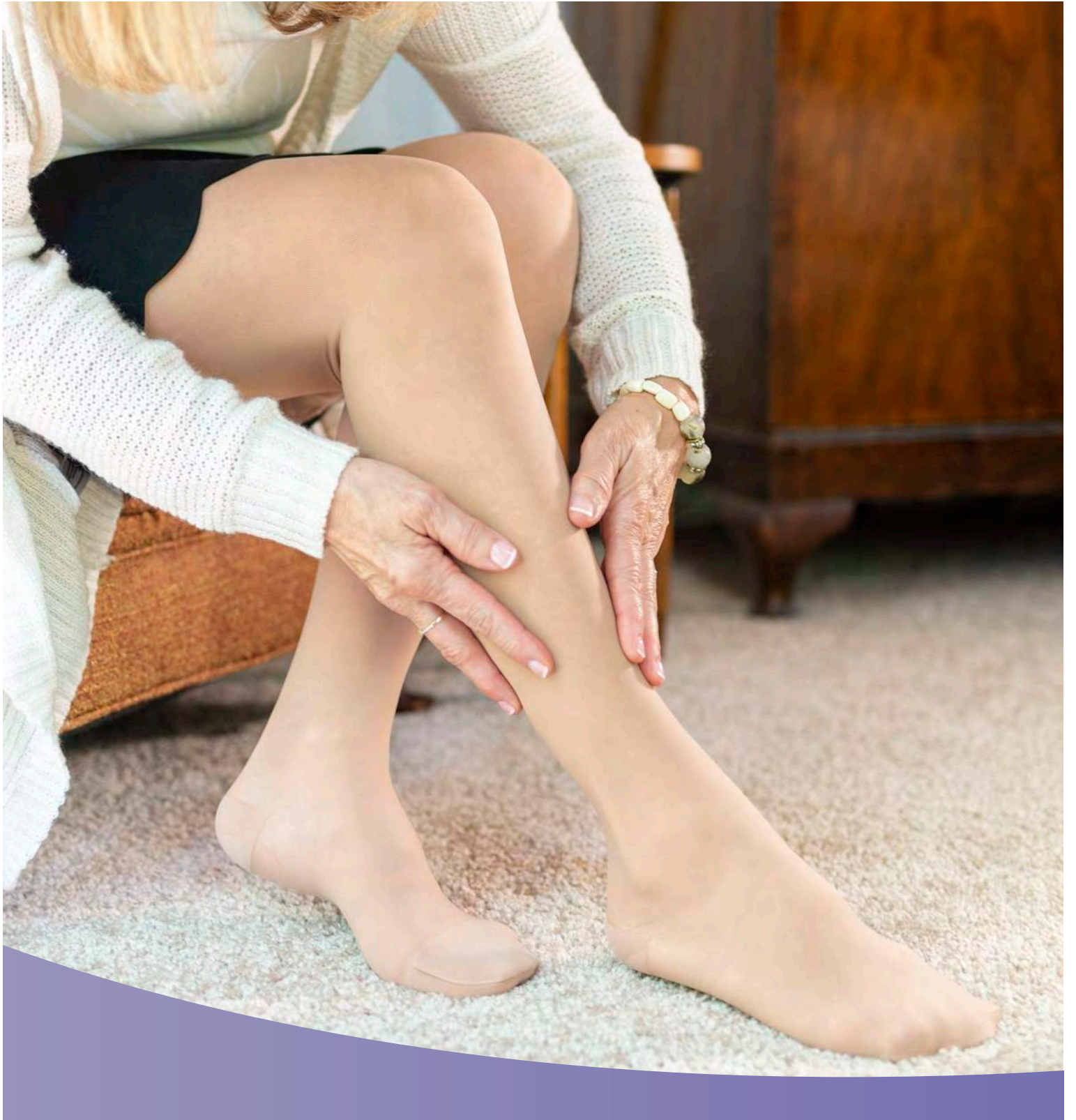

STYLE

Express Yourself

Why choose **Style**?

- Fashionable, modern patterns and designs
- Special occasion and formal wear
- Our most luxurious yarns

Style Products

Product	Series Number	Page
Sea Island Cotton	220	14
Patterns	710	16
Medium Sheer	750	18
Sheer	780	20
Microfiber	820	22
Microfiber Patterns	830	24
Soft Opaque	840	26

220

Style

SEA ISLAND COTTON

for women & men

Style Sea Island Cotton socks are made with the world's rarest cotton, ensuring a luxurious feel that is as soft as cashmere.

Compression Level

20-30mmHg

Also available in 15-20mmHg, see page 54

Features and Benefits



Soft



Cotton



Breathable

- Knitted with the world's rarest cotton for a soft, luxurious feel
- Ideal for sensitive skin
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off

Fiber Content

- 43% Sea Island Cotton, 39% Nylon, 11% Spandex
7% Supima Cotton
- Non-latex

Best Suited for

- Management of symptoms related to Chronic Venous Disease (CVD) ¹
- Patients who prefer natural fibers for everyday wear
- Long-distance travel



20-30mmHg
Closed Toe Calf Large Long in Navy

222CLLM10



20-30mmHg
Closed Toe Calf Medium Long in Black

222CMLM99

Sizing Chart

	Small		Medium		Large		X Large	
Ankle Circumference	7"-8.5" (18-21.5cm)		8.5"-10" (21.5-25.5cm)		10"-11.5" (25.5-29cm)		11.5"-14" (29-36cm)	
Calf Circumference	11"-15.5" (28-39cm)		12.5"-17.5" (31.5-45cm)		14"-20" (35.5-51cm)		16"-24" (41-61cm)	
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Size	SS	SL	MS	ML	LS	LL	XS	XL

Women's Shoe Sizes: Short Leg 5.5-9.5, Long Leg 8-12 Men's Shoe Sizes: Short Leg 7-11, Long Leg 9.5-14

Styles and Colors

Closed Toe = ●

NAVY
10

BLACK
99

Calf (C)	●	●
----------	---	---

Product Code Example: 222CLSW99

Series	mmHg Level	Style	Size	Gender	Color
22	2	C	LS	W	99

710

Style PATTERNS

for women

Style Patterns hosiery is available in fashionable patterns, making it a staple for your professional and everyday wear.

Compression Level

20-30mmHg

Features and Benefits



Sheer



Elegant



Breathable

- Stylish diamond-dot and stripe patterns for professional and dress wear
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off
- Cotton-lined gusset for pantyhose styles

Fiber Content

- 64% Nylon , 36% Spandex
(Content can vary slightly by style)
- Non-latex

Best Suited for

- Management of early stages of Chronic Venous Disease (CVD) ¹
- Pre- and post-vein treatment
- Patients who prefer fashionable options
- Ideal for business or dress




Scalloped
Sensinnov®
Thigh band on
Stripe patterns



Sensinnov
thigh band on
Diamond Dot
patterns

20-30mmHg
Closed Toe Thigh Medium Long in Black

712NMLW99

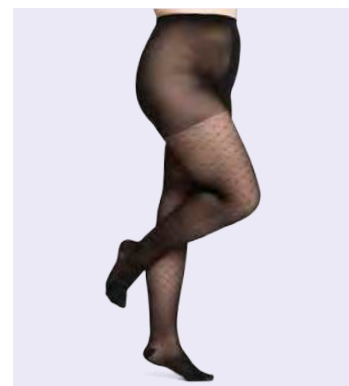

20-30mmHg
Closed Toe Pantyhose Medium Short in Black

712PMSW99

Sizing Chart

	Small		Medium		Large	
Ankle Circumference	7"–8.5" (18–21.5cm)		8.5"–10" (21.5–25.5cm)		10"–11.5" (25.5–29cm)	
Calf Circumference	11"–15.5" (28–39cm)		12.5"–17.5" (31.5–45cm)		14"–20" (35.5–51cm)	
Thigh Circumference	18"–26" (45.5–66cm)		19"–28.5" (48–73cm)		20.5"–33" (52–84cm)	
Leg Length	Short	Long	Short	Long	Short	Long
	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)
Hip Circumference Guidelines	32.5"–44.5" (83–112.5cm)		34.5"–50" (87–127cm)		38.5"–59.5" (98–150.5cm)	
Size	SS	SL	MS	ML	LS	LL

Women's Shoe Sizes Short leg 5.5–9.5, Long leg 8–12

Styles and Colors

Closed Toe = ●	<div>ONYX STRIPE 19</div> <div>BLACK 99</div>	
	●	●
Thigh w/Grip-Top (N) Sensinnov band grip-top	●	●
Pantyhose (P)	●	●

Product Code Example: 712NLSW19

Series	mmHg Level	Style	Size	Gender	Color
71	2	N	LS	W	19

750

Style MEDIUM SHEER

for women

Thanks to the durability of Style Medium Sheer, you will have the confidence to wear sheer hosiery every day.

Compression Level

20-30mmHg

Features and Benefits



Medium Sheer



Durable



Breathable

- Fashionable, durable, medium sheer hosiery for everyday wear
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off
- Cotton-lined gusset for pantyhose styles

Fiber Content

- 68% Nylon, 32% Spandex
(Content can vary slightly by style)
- Non-latex

Best Suited for


- Management of early stages of Chronic Venous Disease (CVD) ¹
- Pre- and post-vein treatment
- Daily wear for business or casual
- Active professionals needing a durable sheer



Silicone
band grip-top
in thigh

20-30mmHg
Closed Toe Thigh Medium Long in Black

752NMLW99

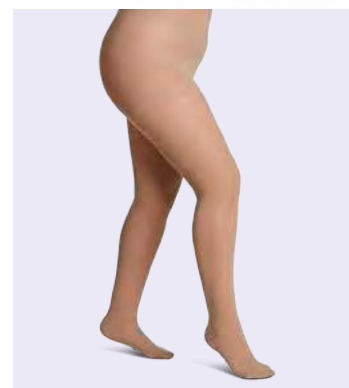

20-30mmHg
Pantyhose Medium Long in Honey

752PMLW33

Sizing Chart

	Small		Medium		Large	
Ankle Circumference	7"–8.5" (18–21.5cm)		8.5"–10" (21.5–25.5cm)		10"–11.5" (25.5–29cm)	
Calf Circumference	11"–15.5" (28–39cm)		12.5"–17.5" (31.5–45cm)		14"–20" (35.5–51cm)	
Calf Length	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	18"–26" (45.5–66cm)		19"–28.5" (48–73cm)		20.5"–33" (52–84cm)	
Leg Length	Short	Long	Short	Long	Short	Long
	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)
Hip Circumference Guidelines	32.5"–44.5" (83–112.5cm)		34.5"–50" (87–127cm)		38.5"–59.5" (98–150.5cm)	
Size	SS	SL	MS	ML	LS	LL

Women's Shoe Sizes Short 5.5–9.5, Long 8–12

Styles and Colors

Closed Toe = ●	<div><div>HONEY 33</div><div>NIGHT-SHADE 94</div><div>BLACK 99</div></div>		
Calf (C)	●	●	●
Thigh w/Grip-Top (N) <small>Silicone band grip-top</small>	●	●	●
Pantyhose (P)	●	●	●

Product Code Example: 752CMLW94

Series	mmHg Level	Style	Size	Gender	Color
75	2	C	ML	W	94

780

Style **SHEER**

for women

Style Sheer hosiery offers fashionable sheerness you can feel comfortable in, making it ideal for dress and professional wear.

Compression Level

15-20mmHg

20-30mmHg

30-40mmHg

Features and Benefits



Sheer



Elegant



Breathable

- Fashionable for professional and dress wear
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off
- Cotton-lined gusset for pantyhose styles

Fiber Content

- 15-20, 20-30mmHg: 69% Nylon , 31% Spandex (content can vary slightly by style)
- 30-40mmHg: 60% Nylon , 40% Spandex (content can vary slightly by style)
- Non-latex

Best Suited for

- Management of early stages of Chronic Venous Disease (CVD) ¹
- Pre- and post-vein treatment
- Patients who prefer fashionable options
- Ideal for business or dress



15-20mmHg
Closed Toe Thigh Medium Long in Dark Navy

781NMLW08



20-30mmHg
Closed Toe Calf Small Long in Mocha

782CSLW85

Sizing Chart

	Small		Medium		Large	
Ankle Circumference	7"–8.5" (18–21.5cm)		8.5"–10" (21.5–25.5cm)		10"–11.5" (25.5–29cm)	
Calf Circumference	11"–15.5" (28–39cm)		12.5"–17.5" (31.5–45cm)		14"–20" (35.5–51cm)	
Calf Length	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	18"–26" (45.5–66cm)		19"–28.5" (48–73cm)		20.5"–33" (52–84cm)	
Leg Length	Short	Long	Short	Long	Short	Long
	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)
Hip Circumference Guidelines	32.5"–44.5" (83–112.5cm)		34.5"–50" (87–127cm)		38.5"–59.5" (98–150.5cm)	
Size	SS	SL	MS	ML	LS	LL

Women's Shoe Sizes Short 5.5–9.5, Long 8–12

Maternity Hip Circumference Guidelines Small ≤56" (142cm), Medium ≤62" (157.5cm), Large ≤70" (178cm)

Styles and Colors

Closed Toe = ●
Open Toe = ○

	DARK NAVY 08	HONEY 33	WARM SAND 34*	GOLDEN 36	TOASTED ALMOND 37*	CAFÉ 73	MOCHA 85	BLACK 99
--	-----------------	-------------	------------------	--------------	-----------------------	------------	-------------	-------------

Calf (C)	●	●○	●○	●○	○○	●○	●○	●○
Thigh w/Grip-Top (N) Sensinnov band grip-top	●	●○	●○	●○	○○	●○	●○	●○
Pantyhose (P)	●	●○	●○	●○	○○	●○	●○	●○
Maternity Pantyhose (M)**		●						●

* 15–20mmHg & 20–30mmHg ONLY

** 20–30mmHg ONLY

Product Code Example: **781PMSW33**

Series	mmHg Level	Style	Size	Gender/Open Toe	Color
78	1	P	MS	W	33

Product Code Example: **783PSSO73**

Series	mmHg Level	Style	Size	Gender/Open Toe	Color
78	3	P	SS	O	73

820

Style

MICROFIBER

for men

Style Microfiber socks offer microfiber softness for your professional and everyday wear.


Compression Level

15-20mmHg

20-30mmHg

30-40mmHg

Features and Benefits

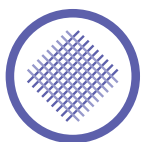

Microfiber



Soft



Stylish

- 100% of microfiber sits next to skin for a soft feel
- Excellent breathability and moisture management for added comfort
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off

Fiber Content

- 87% Nylon , 13% Spandex (Calf)
- 68% Nylon , 32% Spandex (Thigh)
- Non-latex

Best Suited for

- Management of early stages of Chronic Venous Disease (CVD) ¹
- Patients who prefer fashionable options
- Extended foot sizes for men
- Long-distance travel



Silicone
grip-top band
in calf*



20-30mmHg
Closed Toe Calf w/Grip-top Medium Long in
Tan Khaki

822CMLM32/S

Sensinnov
band
grip-top
in thigh



30-40mmHg
Closed Toe Thigh Medium Long in Tan Khaki

823NMLM32

Sizing Chart

	Small		Medium		Large		X Large	
Ankle Circumference	7"–8.5" (18–21.5cm)		8.5"–10" (21.5–25.5cm)		10"–11.5" (25.5–29cm)		11.5"–14" (29–36cm)	
Calf Circumference	11"–15.5" (28–39cm)		12.5"–17.5" (31.5–45cm)		14"–20" (35.5–51cm)		16"–24" (41–61cm)	
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	18"–26" (45.5–66cm)		19"–28.5" (48–73cm)		20.5"–33" (52–84cm)		NA	
Leg Length	Short	Long	Short	Long	Short	Long		
	≤30" (76cm)	>30" (76cm)	≤30" (76cm)	>30" (76cm)	≤30" (76cm)	>30" (76cm)		
Size Shoe size	SS 7–11	SL 9.5–14	MS 7–11	ML 9.5–14	LS 7–11	LL 9.5–14	XS 7–11	XL 9.5–14
				MX* 12.5–17.5		LX* 12.5–17.5		XX* 12.5–17.5

Men's shoe sizes: Short leg 7–11, long leg 9.5–14 (For extended foot sizes: long leg 12.5–17.5)
* MX, LX, and XX extended foot sizes only available in calf style in Black & Tan Khaki in 15–20 & 20–30mmHg only.
Extended foot sizes not available with grip top

Styles and Colors

Closed Toe = ●	<div><div>TAN KHAKI 32</div><div>STEEL GREY 93***</div><div>BLACK 99</div></div>		
Calf (C)	●	●	●
Calf w/Grip-Top (C/S)** <small>Silicone band grip-top</small>	●		●
Thigh w/Grip-Top (N) <small>Sensinnov band grip-top</small>	●		●

** Calf w/ Grip-Top not available in 15–20mmHg
*** Steel Grey color only available in 15–20mmHg and 20–30mmHg

Product Code Example: 822NLSM99

Series	mmHg Level	Style	Size	Gender	Color
82	2	N	LS	M	99

Product Code Example: 822CMLM32/S

Series	mmHg Level	Style	Size	Gender	Color	Grip-top
82	2	C	ML	M	32	/S

830

Style MICROFIBER PATTERNS

for women & men

Style Microfiber Patterns socks combine ultimate comfort with a variety of fashionable colors and designs to spice up your wardrobe.

Compression Level

20–30mmHg

Also available in 15–20mmHg, see page 58

Features and Benefits



Microfiber



Stylish



Patterns

- 100% of microfiber sits next to skin for a soft feel
- Fashionable, durable socks for everyday wear
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off

Fiber Content

- 87% Nylon, 13% Spandex
(content can vary slightly by style)
- Non-latex


Best Suited for

- Management of symptoms related to Chronic Venous Disease (CVD) ¹
- Patients who prefer fashionable options
- Long-distance travel



20–30mmHg
Closed Toe Calf Medium Long in Mariner Stripe

832CMLM02

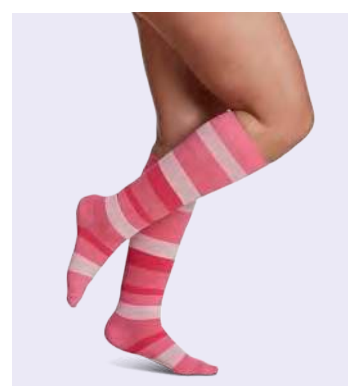

20–30mmHg
Closed Toe Calf Large Short in Pink Stripe


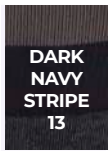

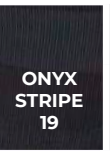






832CLSW15

Sizing Chart

	Small		Medium		Large	
Ankle Circumference	7"-8.5" (18-21.5cm)		8.5"-10" (21.5-25.5cm)		10"-11.5" (25.5-29cm)	
Calf Circumference	11"-15.5" (28-39cm)		12.5"-17.5" (31.5-45cm)		14"-20" (35.5-51cm)	
Calf Length	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Size	SS	SL	MS	ML	LS	LL

Women's Shoe Sizes: Short Leg 5.5-9.5, Long Leg 8-12 Men's Shoe Sizes: Short Leg 7-11, Long Leg 9.5-14

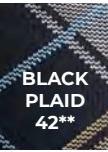Styles and Colors

Closed Toe = ●							
Calf (C)	●	●	●	●	●	●	●
Closed Toe = ●							
Calf (C)	●	●	●	●	●	●	●

* Available for women only
** Available for men only

Product Code Example: 832CMLW22

Series	mmHg Level	Style	Size	Gender	Color
83	2	C	ML	W	22

840

Style SOFT OPAQUE

for women

Style Soft Opaque hosiery blends a fashionable opaque look with long-lasting microfiber softness for your professional and everyday wear.

Compression Level

15-20mmHg


20-30mmHg

30-40mmHg

Features and Benefits



Opaque

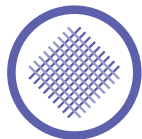

Microfiber



Soft

- Fashionable opaque hosiery for professional and everyday wear
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off
- Cotton-lined gusset for pantyhose styles

Fiber Content


- 74% Nylon , 26% Spandex
(Content can vary slightly by style)
- Non-latex

Best Suited for

- Management of early stages of Chronic Venous Disease (CVD) ¹
- Pre- and post-vein procedure
- Patients who prefer fashionable options
- Pregnancy
- Long distance travel




Silicone
band grip-top
in calf

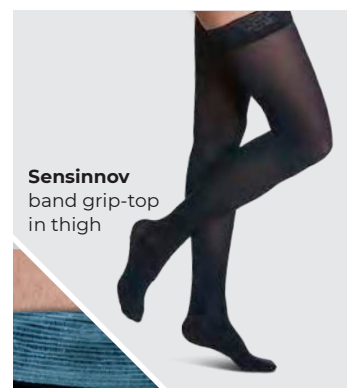

20-30mmHg
Closed Toe Thigh Small Long in Midnight Blue

842NSLW09

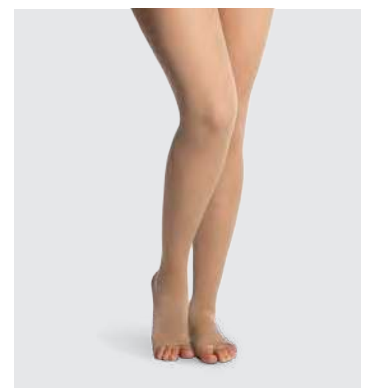

30-40mmHg
Open Toe Pantyhose Large Long in Chai

843PMLO35

Sizing Chart

	Small		Medium		Large	
Ankle Circumference	7"–8.5" (18–21.5cm)		8.5"–10" (21.5–25.5cm)		10"–11.5" (25.5–29cm)	
Calf Circumference	11"–15.5" (28–39cm)		12.5"–17.5" (31.5–45cm)		14"–20" (35.5–51cm)	
Calf Length	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	18"–26" (45.5–66cm)		19"–28.5" (48–73cm)		20.5"–33" (52–84cm)	
Hip Circumference Guidelines	32.5"–44.5" (83–112.5cm)		34.5"–50" (87–127cm)		38.5"–59.5" (98–150.5cm)	
Leg Length	Short	Long	Short	Long	Short	Long
	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)
Size	SS	SL	MS	ML	LS	LL

Women's Shoe Sizes Short leg 5.5–9.5, Long leg 8–12

Maternity hip circumference guidelines Small: <56" (142cm), Medium: <62" (157.5cm), Large: <70" (178cm)

Styles and Colors

Closed Toe = ●
Open Toe = ○

MIDNIGHT BLUE 09	CHAI 35	PECAN 40	MULBERRY 60*	ESPRESSO 89**	GRAPHITE 91	BLACK 99
------------------------	------------	-------------	-----------------	------------------	----------------	-------------

Calf (C)	●	●○	●	●	●	●○
Calf w/Grip-Top (C/S)**						●
Thigh w/Grip-Top (N) Sensinnov band grip-top	●	●○	●	●	●	●○
Pantyhose (P)	●	●○	●	●	●	●○
Maternity Pantyhose (M)*						●

*15–20 & 20–30mmHg ONLY ** 20–30mmHg ONLY

Product Code Example: **842CLLW89**

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color
84	2	C	LL	W	89

sigvaris



MOTION

Push Yourself

Why choose **Motion**?

- Cushioned sole for added comfort
- Ideal for athletic lifestyle
- Comfortable for active professions

Motion Products

Product	Series Number	Page
Cushioned Cotton	360	30
High Tech	412	32
Thermoregulating Wool	422	34

Motion CUSHIONED COTTON

for women & men

Motion Cushioned Cotton is soft and breathable with an extra-padded sole for optimal comfort, and is ideal for sensitive skin.

Compression Level

20–30mmHg

Also available in 15–20mmHg, see page 52

Features and Benefits



**Cushioned
Sole**



Cotton



**Sensitive
Skin**

- Supima cotton for highest level of comfort and breathability
- Extra-padded sole, heel, and toe to protect feet
- Flat, low-friction toe seam

Fiber Content

- 64% Supima Cotton , 28% Nylon , 8% Spandex
- Non-latex

SUPIMA®
WORLD'S FINEST COTTONS

Best Suited for

- Management of symptoms related to Chronic Venous Disease (CVD) ¹
- People who stand for prolonged periods of time
- Work shoes and boots
- Casual shoes and sports shoes



20–30mmHg
Closed Toe Calf Medium Long in White

362CMLM00



20–30mmHg
Closed Toe Calf Small Long in Black

362CSLW99

Sizing Chart

	Small		Medium		Large		X Large	
Ankle Circumference	7"-8.5" (18-21.5cm)		8.5"-10" (21.5-25.5cm)		10"-11.5" (25.5-29cm)		11.5"-14" (29-36cm)	
Calf Circumference	11"-15.5" (28-39cm)		12.5"-17.5" (31.5-45cm)		14"-20" (35.5-51cm)		16"-24" (41-61cm)	
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Size	SS	SL	MS	ML	LS	LL	XS	XL

Women's Shoe Sizes: Short Leg 5.5-9.5, Long Leg 8-12 Men's Shoe Sizes: Short Leg 7-11, Long Leg 9.5-14

Styles and Colors

Closed Toe = ●

WHITE
00

BLACK
99

Calf (C)

●

●

Product Code Example: 362CMLM99

Series	mmHg Level	Style	Size	Gender	Color
36	2	C	ML	M	99

412

Motion HIGH TECH

for women & men

Motion High Tech delivers optimal comfort where it counts with an anatomically-contoured shape and a cushioned sole.

Compression Level

20-30mmHg

Features and Benefits



**Moisture
Wicking**



**Anatomically
Contoured**



Colors


- High-tech fibers for thermal control
- Cushioned footbed, toe, and Achilles tendon zones for optimal comfort
- Additional compression in calf helps improve circulation

Fiber Content

- 64% Nylon, 17% Polyester, 19% Spandex
- Non-latex

Best Suited for

- Management of symptoms related to Chronic Venous Disease (CVD) ¹
- People who stand on their feet all day
- Walking and everyday activities
- Casual and occupational footwear

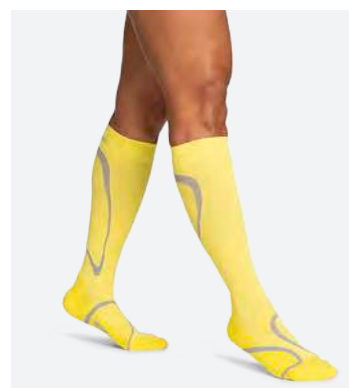

20-30mmHg
Closed Toe Calf Small Medium in Limeade

412CSM64






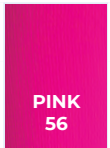





20-30mmHg
Closed Toe Calf Medium Large in Steel Blue

412CML62

Sizing Chart

	Small			Medium			Large			X Large		
Ankle Circumference	7"–8.5" (18–21.5cm)			8.5"–10" (21.5–25.5cm)			10"–11.5" (25.5–29cm)			11.5"–13" (29–33cm)		
Calf Circumference	11"–15.5" (28–39cm)			12.5"–17.5" (31.5–45cm)			14"–20" (35.5–51cm)			16"–23" (40.5–58cm)		
Women's Shoe Size	4.5–7.5	8–10.5	11–13.5	4.5–7.5	8–10.5	11–13.5	8–10.5	11–13.5	>14	8–10.5	11–13.5	>14
Men's Shoe Size	≤6.5	7–9.5	10–12.5	≤6.5	7–9.5	10–12.5	7–9.5	10–12.5	13–15.5	7–9.5	10–12.5	13–15.5
Size	SS	SM	SL	MS	MM	ML	LS	LM	LL	XS	XM	XL

Styles and Colors

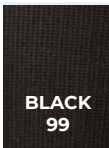Closed Toe = ●									
	WHITE 00	BLUE 50	LIME 54	PINK 56	RED 59	STEEL BLUE 62	BLOOD ORANGE 63	LIMEADE 64	BLACK 99
Calf (C)	●	●	●	●	●	●	●	●	●

Product Code Example: 412CML59

Series	mmHg Level	Style	Size	Color
41	2	C	ML	59

422

Motion THERMOREGULATING WOOL

for women & men

Motion Thermoregulating Wool combines the benefits of natural Merino wool with cushioned soles, making it ideal for wherever the road takes you!

Compression Level

20–30mmHg

Also available in 15–20mmHg, see page 68

Features and Benefits



Wool



**Cushioned Thermoregulating
Sole**



- Natural properties of wool make these socks odor resistant
- Cushioned, padded soles help prevent blisters
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off

Fiber Content

- 67% Fine Merino Wool, 26% Nylon, 7% Spandex
- Non-latex

Best Suited for

- Management of symptoms related to Chronic Venous Disease (CVD) ¹
- People who stand on their feet all day
- Work boots, steel-toed shoes, and other occupational footwear



20–30mmHg
Closed Toe Calf Large Long in Charcoal

422CLL12



20–30mmHg
Closed Toe Calf Small Long in Olive

422CSL31

Sizing Chart

	Small			Medium			Large		
Ankle Circumference	7"-8.5" (18-21.5cm)			8.5"-10" (21.5-25.5cm)			10"-11.5" (25.5-29cm)		
Calf Circumference	11"-15.5" (28-39cm)			12.5"-17.5" (31.5-45cm)			14"-20" (35.5-51cm)		
Women's Shoe Size	4.5-7.5	8-10.5	11-13.5	4.5-7.5	8-10.5	11-13.5	8-10.5	11-13.5	>14
Men's Shoe Size	≤6.5	7-9.5	10-12.5	≤6.5	7-9.5	10-12.5	7-9.5	10-12.5	13-15.5
Size	SS	SM	SL	MS	MM	ML	LS	LM	LL

Styles and Colors

Closed Toe = ●

CHARCOAL
12

OLIVE
31

Calf (C)	●	●
----------	---	---

Product Code Example: 422CLL31

Series	mmHg Level	Style	Size	Color
42	2	C	LL	31

sigvaris



SPECIALTY

Take care of Yourself

Why choose **Specialty**?
Created for specific medical needs, including:

- Venous conditions
- Wound care management
- Lymphatic support
- Diabetes needs

Specialty Products		
Product	Series Number	Page
Natural Rubber	500	38
Secure	550	40
Secure Arm	560	42
Diabetic Compression Sock	602	44
Advance Armsleeve	910	46

500

Specialty

NATURAL RUBBER

for women & men

Natural Rubber offers patients a durable, long-term containment solution for compression therapy with minimal fiber fatigue throughout the day.

Compression Level

30–40mmHg

40–50mmHg

50–60mmHg

Features and Benefits



Natural Rubber



Durable



Wide Size Range

- Ideal for patients with more serious indications
- Extended sizing for a custom-like fit

Fiber Content

- 70% Nylon, 30% Natural Latex Rubber (Calf & Thigh styles)
- 75% Nylon, 20% Natural Latex Rubber, 5% other fibers (Pantyhose style)
- **Caution:** contains natural rubber latex that may cause allergic reaction

Best Suited for

- Varicosities¹
- Chronic edema
- Venous stasis
- Early stages of lymphedema
- Active venous ulcers



Silicone
band grip-top
in thigh



30–40mmHg
Open Toe Thigh Small Average Long in Beige

503NS2077



40–50mmHg
Open Toe Calf Medium Full Short in Beige

504CM3077

Sizing Chart

	Small				Medium				Large				X Large			
Ankle Circumference	7.5"–9" (19–23cm)				9"–12" (23–30cm)				12"–15.5" (30–40cm)				15.5"–18" (40–46cm)			
Calf Circumference	Average		Full		Average		Full		Average		Full		Average		Full	
	11"–15" (28–38cm)		≤17" (≤43cm)		12.5"–17" (32–43cm)		≤19" (≤48cm)		14.5"–19" (36–48cm)		≤22.5" (≤57cm)		18"–22.5" (45–57cm)		≤27" (≤68cm)	
Calf Circumference for 505C (Full only)	505C: full calf only		12.5"–17" (32–43cm)		505C: full calf only		14.5"–19" (36–48cm)		505C: full calf only		18"–22.5" (45–57cm)		505C: not available in XL			
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long
	≤15.5" (40cm)	>15.5" (40cm)	≤15.5" (40cm)	>15.5" (40cm)	≤15.5" (40cm)	>15.5" (40cm)	≤15.5" (40cm)	>15.5" (40cm)	≤15.5" (40cm)	>15.5" (40cm)	≤15.5" (40cm)	>15.5" (40cm)	≤15.5" (40cm)	>15.5" (40cm)	≤15.5" (40cm)	>15.5" (40cm)
Thigh Circumference	Average		Full		Average		Full		Average		Full		Custom sizes available. Please contact customer care for details.			
	16"–21" (41–54cm)		≤24" (61cm)		19"–24" (48–61cm)		≤28" (71cm)		23"–28" (58–71cm)		≤32" (81cm)					
Thigh Circumference for 505T (Full only)	505T: full only		≤24" (61cm)		505T: full only		≤28" (71cm)		505T: full only		≤32" (81cm)					
Leg Length	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long				
	≤28" (71cm)	>28" (71cm)	≤28" (71cm)	>28" (71cm)	≤28" (71cm)	>28" (71cm)	≤28" (71cm)	>28" (71cm)	≤28" (71cm)	>28" (71cm)	≤28" (71cm)	>28" (71cm)				
Hip Circumference Guidelines	≤48" (122cm)				≤54" (137cm)				≤61" (155cm)							
Size	S1	S2	S3	S4	M1	M2	M3	M4	L1	L2	L3	L4	X1	X2	X3	X4

Styles and Colors

Open Toe = O



Caution: this product contains natural rubber latex which may cause allergic reactions. For all indications, cautions, and contraindications, please see page 96 for details. Sigvaris accepts no liability for non-observance of contraindications or cautions.

Calf (C)	O	Calf only available in S, M, and L full circumference for 50-60mmHg
Thigh w/out Grip-Top (T)	O	
Thigh w/Grip-Top (N) Silicone band grip-top	O	Not available in 50-60mmHg
Pantyhose (P)	O	Pantyhose only available in 30-40mmHg and only in full thigh circumference
Thigh w/Waist Attachment (W) Indicate leg LEFT (/L) or RIGHT (/R)	O	Thigh w/ waist attachment is available in average and full thigh circumferences in 30-40mmHg and in full thigh circumferences only in 40-50mmHg

Product Code Example: 504CS2077

Series	mmHg Level	Style	Size	Gender/Open Toe	Color
50	4	C	S2	O	77

Product Code Example: 503WL1077/L

Series	mmHg Level	Style	Size	Gender/Open Toe	Color	Leg
50	3	W	L1	O	77	/L

Specialty

SECURE

for women & men

Secure combines compression and containment with comfort and durability that's optimal for comfortably managing advanced venous edema, lymphedema, and wound care.

Compression Level

20-30mmHg

30-40mmHg

40-50mmHg*

Features and Benefits



Containment



**Dotted
Grip-top Band**



Opaque

- More graduated compression and containment than other traditional garments
- Durable double-covered inlay yarns allow comfortable all-day wear

Fiber Content

- 76% Nylon, 24% Spandex (Thigh style)
- Non-latex

Indications

- Chronic Venous Insufficiency (CVI) ¹
- Lymphedema that is well-managed
- Lipedema
- Varicosities
- Deep vein thrombosis (DVT)
- Superficial phlebitis
- Post-thrombotic syndrome
- Orthostatic hypotension



**Silicone
band grip-top**

30-40mmHg
Closed Toe Calf Large Long in Black

553CL2M99/S



40-50mmHg
Closed Toe Thigh Medium Full Long in Beige

554NM4W77

* 40-50mmHg products made on demand.


Sizing Chart

	Medium				Large				X Large				XX Large			
Ankle Circumference	8.5"-10" (21.5-25.5cm)				10"-11.5" (25.5-29cm)				11.5"-13" (29-33cm)				13"-14.5" (33-37cm)			
Calf Circumference	Average		Full		Average		Full		Average		Full		Average		Full	
	12"-17.5" (31-45cm)		17.5"-23" (45-59cm)		13.5"-19" (35-49cm)		19"-25" (49-63cm)		16"-22" (41-56cm)		22"-28" (56-71cm)		17.5"-23" (45-59cm)		23"-29" (59-74cm)	
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Thigh Circumference	Average		Full		Average		Full		Average		Full		Average		Full	
	20"-27" (51-69cm)		23.5"-31" (60-78cm)		24"-30" (61-77cm)		26.5"-34" (68-86cm)		27"-33.5" (69-85cm)		30"-37" (76-94cm)		29.5"-36.5" (75-93cm)		33"-40" (84-102cm)	
Leg Length	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long
	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)	≤28.5" (72cm)	>28.5" (72cm)
Size	M1	M2	M3	M4	L1	L2	L3	L4	X1	X2	X3	X4	G1	G2	G3	G4

Women's Shoe Sizes: Short Leg 5.5-9.5, Long Leg 8-12 Men's Shoe Sizes: Short Leg 7-11, Long Leg 9.5-14

Styles and Colors

Closed Toe = ●
Open Toe = ○

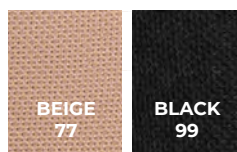

Calf w/Grip-Top (C/S) Silicone band grip-top	●○	●●
Thigh w/Grip-Top (N) Silicone band grip-top	●○	●●

Product Code Example: **552CM2W77/S**

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color	Grip-top
55	2	C	M2	W	77	/S

Specialty SECURE ARMSLEEVE, GLOVE, AND GAUNTLET for women & men

The Secure Armsleeve combines compression and containment with comfort and durability that's optimal for managing lymphedema.

Compression Level

15-20mmHg

20-30mmHg

30-40mmHg

Features and Benefits



Containment



**Dotted
Grip-top Band**



Colors


- More graduated compression and containment than traditional garments
- Full-fingered Glove and Gauntlet available
- Provides UPF 50+ sun protection

Fiber Content

- 78% Nylon, 22% Spandex (Armsleeve)
- 60% Nylon, 40% Spandex (Glove, Gauntlet)
- Non-latex

Indications

- Chronic edema
- Lymphedema that is well-managed
- Post-operative edema
- Post-mastectomy care

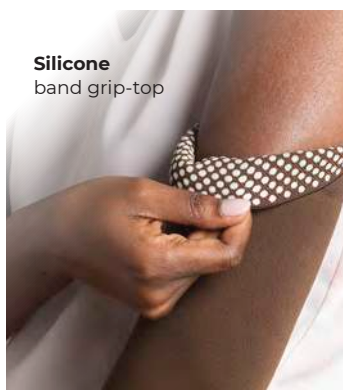

Silicone
band grip-top

30-40mmHg
Armsleeve Medium Short Average in Cocoa

563AM184



20-30mmHg
Gauntlet Medium in Beige


562GM78

Sizing Chart Arm

	Small				Medium				Large				X Large				XX Large			
Wrist	5.5"-6.5" (14-16cm)				6.5"-7" (16-18cm)				7"-8" (18-20cm)				8"-9" (20-23cm)				9"-10" (23-26cm)			
Elbow	Average		Full		Average		Full		Average		Full		Average		Full		Average		Full	
	8"-10" 20-26cm		10"-13.5" 26-34cm		9"-11.5" 23-29cm		11.5"-14.5" 29-37cm		10"-12.5" 26-32cm		12.5"-15.5" 32-40cm		11.5"-13.5" 29-34cm		13.5"-16.5" 34-42cm		12.5"-14.5" 32-37cm		14.5"-17.5" 37-45cm	
Upper Arm	Average		Full		Average		Full		Average		Full		Average		Full		Average		Full	
	10"-13.5" 26-34cm		13.5"-15.5" 34-40cm		11.5"-14.5" 29-37cm		14.5"-17" 37-43cm		12.5"-15.5" 32-40cm		15.5"-18" 40-46cm		13.5"-16.5" 34-42cm		16.5"-19" 42-48cm		14.5"-17.5" 37-45cm		17.5"-20.5" 45-52cm	
Arm Length	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long	Short	Long
	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)
Size	S1	S2	S3	S4	M1	M2	M3	M4	L1	L2	L3	L4	X1	X2	X3	X4	G1	G2	G3	G4

Sizing Chart Hand

	X Small		Small		Medium		Large		X Large		XX Large	
Palm Circumference	6"-6.5" (16-17cm)		7"-7.5" (18-19cm)		8"-8.5" (20-21cm)		8.5"-9" (22-23cm)		9.5"-10" (24-25cm)		10.5"-11" (26-28cm)	
Size	XS		S		M		L		XL		G	

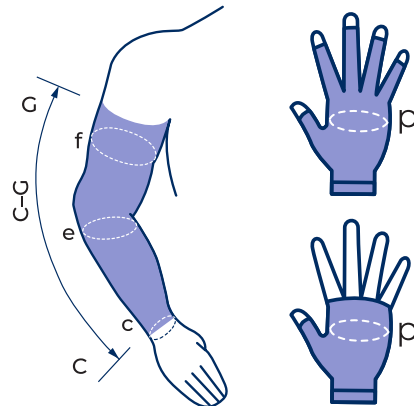

Armsleeve

Product Code Example: **563AL265**

Series	mmHg Level	Style	Size	Color
56	3	A	L2	65

Glove

Product Code Example: **562FM78**

Series	mmHg Level	Style	Size	Color
56	2	F	M	78

Gauntlet

Product Code Example: **562GM78**

Series	mmHg Level	Style	Size	Color
56	2	G	M	78

Styles and Colors

Sleeve, Glove, and Gauntlet = ●	DUSTY ROSE 65	BEIGE 78	COCOA 84	BLACK 99
Armsleeve (A)	●	●	●	●
Glove (F)*	●	●	●	●
Gauntlet (G)*	●	●	●	●

* Glove and Gauntlet available in 15-20mmHg & 20-30mmHg

Specialty

DIABETIC COMPRESSION SOCKS for women & men

Diabetic Compression Socks are the only patented, clinically-proven graduated compression socks constructed for people with diabetes.

Compression Level

18-25mmHg

Also available in 8-15mmHg, see page 70

Features and Benefits



Diabetes



**Moisture
Wicking**



**Cushioned
Sole**

- Padded soles and invisible toe seam add comfort and protection
- Compression-free foot for optimum comfort
- Double-covered inlay yarns provide comfort and allow product to easily glide on and off
- *drirelease*® yarns add moisture wicking properties

Fiber Content

- 76% *drirelease* (65% Polyester, 11% Cotton), 19% Nylon, 5% Spandex
- Non-latex

Best Suited for

- Individuals with diabetes and symptoms related to Chronic Venous Disease (CVD) ¹
- Individuals with peripheral neuropathy



18-25mmHg
Closed Toe Calf Medium Long in White

602CMLM00



18-25mmHg
Closed Toe Calf Small Short in White

602CSSW00

Sizing Chart

	Small		Medium		Large		X Large	
Ankle Circumference	7"-8.5" (18-21.5cm)		8.5"-10" (21.5-25.5cm)		10"-11.5" (25.5-29cm)		11.5"-14" (29-36cm)	
Calf Circumference	11"-15.5" (28-39cm)		12.5"-17.5" (31.5-45cm)		14"-20" (35.5-51cm)		16"-24" (41-61cm)	
Calf Length	Short	Long	Short	Long	Short	Long	Short	Long
	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)	≤16" (41cm)	>16" (41cm)
Size	SS	SL	MS	ML	LS	LL	XS*	XL*

Women's Shoe Sizes: Short Leg 5.5-9.5, Long Leg 8-12 Men's Shoe Sizes: Short Leg 7-11, Long Leg 9.5-14

*XS & XL Only Available In Men's Sizing

Styles and Colors

Closed Toe = ●



Calf (C)



Product Code Example: **602CMLW00**

Series	mmHg Level	Style	Size	Gender/ Open Toe	Color
60	2	C	ML	W	00

Specialty ADVANCE ARMSLEEVE

for women & men

The Advance Armsleeve combines ultimate softness with silver ion properties and a Sensinnov grip-top band that's ideal for all-day comfort and effectiveness.

Compression Level

15-20mmHg

20-30mmHg

30-40mmHg

Features and Benefits



**Moisture
Wicking**



**Sensinnov
Grip-top
Band**



Breathable

- Ideal for patients with arm lymphedema or post-mastectomy care
- Sensinnov top-band ensures armsleeve stays in place all day long
- Provides moisture wicking properties to keep skin dry and comfortable
- Breathable to provide comfortable, year-round wear

Fiber Content

- 50% Nylon, 28% Rayon, 22% Spandex (Content can vary slightly by style)
- Non-latex

Best Suited for

- Chronic edema ¹
- Early stages of lymphedema
- Venous ulcers



15-20mmHg
Armsleeve Without Gauntlet
Small Long in Beige

911ASLO77/S



20-30mmHg
Armsleeve With Gauntlet
Small Long in Beige

912ASLG77/S

Sizing Chart

	Small		Medium				Large				X Large			
Palm	6.5"-8" (17-21cm)		7"-8.5" (18-22cm)				7.5"-9" (19-23cm)				8"-9.5" (20-24cm)			
Wrist	5.5"-6.5" (14-16cm)		6.5"-7" (16-18cm)				7"-8" (18-20cm)				8"-9" (20-23cm)			
Upper Mid-Arm	Regular		Regular		Plus		Regular		Plus		Regular		Plus	
	10"-13.5" 26-34cm		11.5"-14.5" 29-37cm		14.5"-17" 37-43cm		12.5"-15.5" 32-40cm		15.5"-18" 40-46cm		13.5"-16.5" 34-42cm		16.5"-19" 42-48cm	
Arm Length	Regular	Long	Regular	Long	Regular	Long	Regular	Long	Regular	Long	Regular	Long	Regular	Long
	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)	≤17" (43cm)	>17" (43cm)
Size	SR	SL	MR	ML	MR/P	ML/P	LR	LL	LR/P	LL/P	XR	XL	XR/P	XL/P

Styles and Colors

Sleeve = ●




Armsleeve w/Gauntlet (G)

Sensinnov band grip-top



Armsleeve w/out Gauntlet (O)

Sensinnov band grip-top

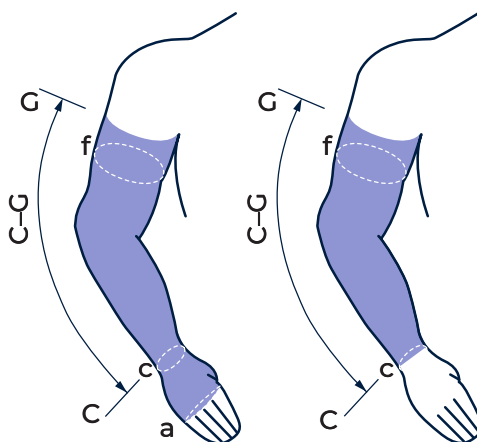**Note:** all armsleeves come with Grip-top (/S)15-20mmHg not available with gauntlet
and not available in plus sizing

Armsleeve without gauntlet

Product Code Example: **912ASLO77/S**

Series	mmHg Level	Style	Size	W/out Gauntlet	Color	Grip-Top
91	2	A	SL	O	77	/S

Armsleeve with gauntlet

Product Code Example: **912ALRG77/PS**

Series	mmHg Level	Style	Size	W/ Gauntlet	Color	Plus Size	Grip-Top
91	2	A	LR	G	77	/P	S



WELL BEING

Care for Yourself

Why choose **WELL BEING**?

- Tired, heavy, restless, aching legs
- Travelers, occupational leg health risks, expecting mothers, family history of chronic venous disease
- Prevention of swelling and varicose veins²
- Simplified sizing

WELL BEING Products

Product	Series Number	Page
CASUAL COTTON	146 / 186	50
CUSHIONED COTTON	142 / 182	52
SEA ISLAND COTTON	151 / 191	54
ALL-SEASON MERINO WOOL	152 / 192	56
MICROFIBER SHADES	143 / 183	58
SHEER FASHION	120	60
SOFT SILHOUETTE LEGGINGS	170	62
ATHLETIC RECOVERY SOCKS	401	64
PERFORMANCE SLEEVES	412	66
MERINO OUTDOOR SOCKS	421	68
EVERSOFT DIABETIC SOCKS	160	70

146/186

CASUAL COTTON

for women & men

Compression Level

15-20mmHg

Features and Benefits

- » Made with premium Supima Cotton for optimum softness and durability
- » Flat, low-friction toe seam to prevent irritation
- » Soft, stretchable, knit-in band prevents pinching

Fiber Content

- » 66% Supima Cotton, 26% Nylon, 8% Spandex
- » Non-latex


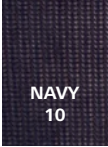
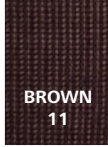




Best Suited for

- » Tired, heavy, restless, aching legs
- » Travelers, occupational leg health risks, expecting mothers, family history of chronic venous disease



SUPIMA®
WORLD'S FINEST COTTONS

Sizing Chart				
146	Women's Shoe	5-7	7.5-9.5	10-12
	Ankle Circumference	7"-9" (18-23cm)	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)
186	Men's Shoe	6.5-8.5	9-11	11.5-14
	Ankle Circumference	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)	8.5"-11.5" (21-29cm)
Size		A	B	C

Style and Colors					
Closed Toe = ●					
		WHITE 00	NAVY 10	BROWN 11	BLACK 99
Calf (C)					

Product Code Example: **146CB10**

Series	Style	Size	Color
146	C	B	10

146
Women's Calf
Style



15-20mmHg
Closed Toe Calf Size B in Khaki

146CB30

186
Men's Calf
Style



15-20mmHg
Closed Toe Calf Size C in Black

186CC99

142/182

CUSHIONED COTTON

for women & men

Compression Level

15–20mmHg

Also available in 20–30mmHg, see page 30

Features and Benefits

- » Cushioned sole, heel, and toe area to protect feet
- » Flat, low-friction toe seam
- » Soft, stretchable knit-in band prevents pinching
- » Ideal for everyday wear
- » Ideal for sensitive skin
- » No compression in heel and instep for ease of donning

Fiber Content

- » 68% Supima Cotton, 25% Nylon, 7% Spandex
- » Non-latex

Best Suited for

- » Tired, heavy, restless, aching legs
- » Travelers, occupational leg health risks, expecting mothers, family history of chronic venous disease
- » Sensitive skin



SUPIMA®
WORLD'S FINEST COTTONS

Sizing Chart				
142	Women's Shoe	5-7	7.5-9.5	10-12
	Ankle Circumference	7"-9" (18-23cm)	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)
182	Men's Shoe	6.5-8.5	9-11	11.5-14
	Ankle Circumference	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)	8.5"-11.5" (21-29cm)
Size		A	B	C

Style and Colors			
Closed Toe = ●			
<div><div>WHITE 00</div><div>COYOTE BROWN 86</div><div>BLACK 99</div></div>			
Calf (C)	●	●	●

Product Code Example: **142CB00**

Series	Style	Size	Color
142	C	B	00



15-20mmHg
Closed Toe Calf Size B in White

142CB00



15-20mmHg
Closed Toe Calf Size C in Black

182CC99

151/191

SEA ISLAND COTTON

for women & men

Compression Level

15–20mmHg

Also available in 20–30mmHg, see page 14

Features and Benefits

- » Knitted from the world's rarest cotton
- » Sea Island Cotton yarns are lightweight and have great strength, a silky luster, and an incredibly soft feel
- » Breathable and comfortable for sensitive skin
- » Ideal for travel and everyday wear

Fiber Content

- » 43% Sea Island Cotton, 39% Nylon, 11% Spandex, 7% Supima Cotton
- » Non-latex

Best Suited for

- » Tired, heavy, restless, aching legs
- » Travelers, occupational leg health risks, expecting mothers, family history of chronic venous disease



Sizing Chart				
151	Women's Shoe	5-7	7.5-9.5	10-12
	Ankle Circumference	7"-9" (18-23cm)	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)
191	Men's Shoe	6.5-8.5	9-11	11.5-14
	Ankle Circumference	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)	8.5"-11.5" (21-29cm)
Size		A	B	C

Style and Colors		
Closed Toe = ●	<div>NAVY 10</div> <div>BLACK 99</div>	
	Calf (C)	<div>●</div> <div>●</div>

Product Code Example: **191CC10**

Series	Style	Size	Color
191	C	C	10



15-20mmHg
Closed Toe Calf Size A in Navy

151CA10



15-20mmHg
Closed Toe Calf Size B in Black

191CB99

152/192

ALL-SEASON MERINO WOOL

for women & men

Compression Level

15-20mmHg

Features and Benefits

- » Soft and comfortable
- » Natural properties of wool make these socks odor resistant
- » Thermoregulating for year-round wear and keeps feet dry

Fiber Content

- » 53% Extra-fine Merino Wool, 38% Nylon, 9% Spandex
- » Non-latex

Best Suited for

- » Tired, heavy, restless, aching legs
- » Travelers, occupational leg health risks, expecting mothers, family history of chronic venous disease



Sizing Chart				
152	Women's Shoe	5-7	7.5-9.5	10-12
	Ankle Circumference	7"-9" (18-23cm)	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)
192	Men's Shoe	6.5-8.5	9-11	11.5-14
	Ankle Circumference	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)	8.5"-11.5" (21-29cm)
Size		A	B	C

Style and Colors			
Closed Toe = ●			
	<div>NAVY 10</div>	<div>BROWN 11</div>	<div>BLACK 99</div>
Calf (C)	●	●	●

Product Code Example: 192CB10

Series	Style	Size	Color
192	C	B	10



15-20mmHg
Closed Toe Calf Size A in Black

152CA99



15-20mmHg
Closed Toe Calf Size C in Brown

192CC11

MICROFIBER SHADES

for women & men

Compression Level

15-20mmHg

Also available in 20-30mmHg, see page 24

Features and Benefits

- » Advanced microfiber technology for ultimate comfort
- » Glides on and off with ease
- » Excellent breathability and moisture management
- » Soft and durable sock for everyday wear

Fiber Content













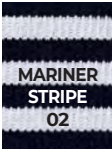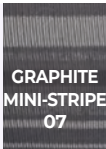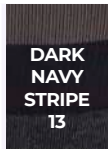

- » 87% Nylon, 13% Spandex
- » Non-latex

Best Suited for

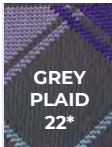- » Tired, heavy, restless, aching legs
- » Travelers, occupational leg health risks, expecting mothers, family history of chronic venous disease



Sizing Chart				
143	Women's Shoe	5-7	7.5-9.5	10-12
	Ankle Circumference	7"-9" (18-23cm)	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)
183	Men's Shoe	6.5-8.5	9-11	11.5-14
	Ankle Circumference	7.5"-10" (19-25cm)	8"-10.5" (20-27cm)	8.5"-11.5" (21-29cm)
Size		A	B	C

Style and Colors								
Closed Toe = ● <i>Actual color may vary</i>								
	Calf (C)	●	●	●	●	●	●	●
Closed Toe = ● <i>Actual color may vary</i>								
	Calf (C)	●	●	●	●	●	●	●

* Available for women only
** Available for men only

Product Code Example: 143CB02			
Series	Style	Size	Color
143	C	B	02



15-20mmHg
Closed Toe Calf Size B in Royal Blue Argyle

143CB48



15-20mmHg
Closed Toe Calf Size C in Black Plaid

183CC42

SHEER FASHION

for women

Compression Level

15-20mmHg

Features and Benefits

- » Fabric feels soft against the skin
- » Incredible sheer design in a variety of fashionable colors
- » Sheer toe and heel area
- » Comfortable stretch is excellent for everyday wear
- » Cotton-lined gusset on pantyhose styles

Fiber Content

- » 84% Nylon, 16% Spandex (Calf style)
- » Content can vary by style
- » Non-latex


Best Suited for

- » Tired, heavy, restless, aching legs
- » Travelers, occupational leg health risks, expecting mothers, family history of chronic venous disease



15-20mmHg
Closed Toe Calf Size C in Charcoal

120CC12

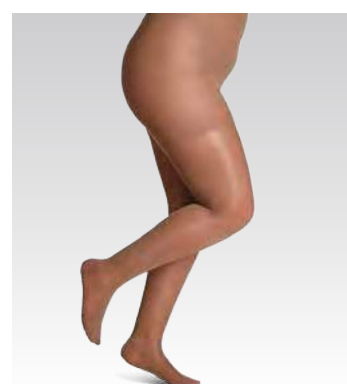

15-20mmHg
Closed Toe Pantyhose Size F in Café

120PF73

Calf / Thigh Sizing Chart				
120	Women's Shoe	5-7	7.5-9.5	10-12
	Ankle Circumference	7.5"-9" (19-23cm)	8"-10" (20.5-25cm)	8.5"-10.5" (22-27cm)
	Size	A	B	C

Weight Guidelines For Thigh Style: Size A: <135Lbs., Size B: <180Lbs., Size C: <230Lbs.

Pantyhose Sizing Chart

Ft/Cm	Lbs/Kg	100 / 45	105 / 48	110 / 50	115 / 52	120 / 54	125 / 57	130 / 59	135 / 61	140 / 64	145 / 66	150 / 68	155 / 70	160 / 73	165 / 75	170 / 77	175 / 80	180 / 82	185 / 84	190 / 86	195 / 89	200 / 91	205 / 93	210 / 95	215 / 98	220 / 100	225 / 102	230 / 105
4'9" / 145cm																												
4'10" / 147cm																												
4'11" / 150cm																												
5'0" / 152cm																												
5'1" / 155cm																												
5'2" / 157cm																												
5'3" / 160cm																												
5'4" / 162cm																												
5'5" / 165cm																												
5'6" / 167cm																												
5'7" / 170cm																												
5'8" / 173cm																												
5'9" / 175cm																												
5'10" / 178cm																												
5'11" / 180cm																												
6' / 183cm																												

Styles and Colors						
Closed Toe = ● Open Toe = ○						
	CHARCOAL 12	TAUPE 29	HONEY 33	GOLDEN 36	CAFÉ 73	BLACK 99
Calf (C)	●	○○	○○	○○	●	●
Thigh (N)	●	●	●	●	●	●
Pantyhose (P)	●	●	●	●	●	●

Product Code Example: 120NC99

Series	Style	Size	Color
120	N	C	99

Product Code Example: 120CCO36

Series	Style	Size	Open Toe	Color
120	C	C	O	36

SOFT SILHOUETTE LEGGINGS

for women

Compression Level

15-20mmHg

Features and Benefits


- » Stylish, subtle ribbed design looks chic and pairs perfectly with dresses, blouses, tunics, and shirts
- » Foldable, comfort-stretch waistband provides a seamless look for all-day wearing comfort
- » Advanced soft stretch microfiber fabric shapes legs and smooths skin

Fiber Content

- » 83% Nylon, 17% Spandex
- » Non-latex

Best Suited for

- » Tired, heavy, restless, aching legs
- » Occupational leg health risks, expecting mothers, family history of chronic vein disease

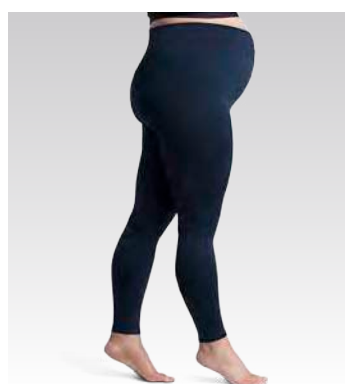

15-20mmHg
Maternity Leggings Size D in Midnight Blue

170MD09



15-20mmHg
Leggings Size A in Mulberry

170LA60

Sizing Chart

	Lbs/Kg	100 / 45	105 / 48	110 / 50	115 / 52	120 / 54	125 / 57	130 / 59	135 / 61	140 / 64	145 / 66	150 / 68	155 / 70	160 / 73	165 / 75	170 / 77	175 / 80	180 / 82	185 / 84	190 / 86	195 / 89	200 / 91	205 / 93	210 / 95	215 / 98	220 / 100	225 / 102	230 / 105
Ft/cm																												
4'10" / 147cm																												
4'11" / 150cm																												
5'0" / 152cm																												
5'1" / 155cm																												
5'2" / 157cm																												
5'3" / 160cm																												
5'4" / 162cm																												
5'5" / 165cm																												
5'6" / 167cm																												
5'7" / 170cm																												
5'8" / 173cm																												
5'9" / 175cm																												
5'10" / 178cm																												
5'11" / 180cm																												
6' / 183cm																												

Guidelines for Ankle Circumference
Size A: 7"-9" (18-23cm), Size B & C: 8"-10" (20-26cm), Size D: 8.5"-11.5" (22-29cm)
Note: Please use pre-pregnancy weight for maternity leggings

Styles and Colors				
Open Foot = ●	MIDNIGHT BLUE 09*	MULBERRY 60*	ESPRESSO 89*	BLACK 99
Leggings (L)	●	●	●	●
Maternity (M)*	●	●	●	●

* Maternity style, Midnight Blue, Mulberry, and Espresso colors are made to order and shipped in 7-10 business days



Non-binding ankle cuff stays comfortably in place for all day wear

Product Code Example: 170LB99			
Series	Style	Size	Color
170	L	B	99

Product Code Example: 170MD09			
Series	Style	Size	Color
170	M	D	09

ATHLETIC RECOVERY SOCKS

for women & men

Compression Level

15–20mmHg

Features and Benefits

- » Helps reduce lactic acid buildup³
- » Comfortable and easy to put on and take off
- » drirelease yarns provide moisture wicking and odor control
- » No compression in heel and instep for ease of donning

Fiber Content

- » 67% drirelease (57% Polyester, 10% Cotton), 26% Nylon, 7% Spandex
- » Non-latex

Best Suited for

- » Tired, heavy, restless, aching legs
- » Post-workout recovery
- » Mild athletic activity



Sizing Chart						
401	Women's Shoe	<7.5	7.5–9.5	10–12	12.5–15	≥15.5
	Men's Shoe	<6.5	6.5–8.5	9–11	11.5–14	14.5–17
	Ankle Circumference	7"–9.5" (18–24cm)	7.5"–10" (19–26cm)	8"–10.5" (20–27cm)	8.5"–11.5" (21–29cm)	8.5"–11.5" (21–29cm)
	Size	S	M	L	X	2

Style and Colors

Closed Toe = ●

WHITE
00

BLACK
99

Calf (C)

●

●

Product Code Example: **401CL00**

Series	Style	Size	Color
401	C	L	00



15-20mmHg
Closed Toe Calf Size Medium in Black

401CM99



15-20mmHg
Closed Toe Calf Size Small in White

401CS00

PERFORMANCE SLEEVES

for women & men

Compression Level

20–30mmHg

Features and Benefits

- » Fabric resists deterioration in salt, chlorine, or fresh water
- » Helps reduce lactic acid buildup³
- » Quick drying
- » High-visibility reflective transfer

Fiber Content

- » 75% Nylon, 25% Spandex
- » Non-latex

Best Suited for

- » Running, triathlons, endurance sports, and gym workouts
- » Team sports
- » Occupational wear



Sizing Chart					
412	Ankle Circumference	7"–8.5" (18–21.5cm)	8.5"–10" (21.5–25.5cm)	10"–11.5" (25.5–29cm)	11.5"–13" (29–33cm)
	Calf Circumference	11"–15.5" (28–39cm)	12.5"–17.5" (31.5–45cm)	14"–20" (35.5–51cm)	16"–23" (40.5–58cm)
	Size	S	M	L	X

Style and Colors					
Calf Sleeve = ●		<div>WHITE 00</div>	<div>BLUE 50</div>	<div>PINK 56*</div>	<div>PURPLE 58*</div>
Calf Sleeve (V)		<div>○</div>	<div>●</div>	<div>●</div>	<div>●</div>

* Pink and purple not available in X-Large

Product Code Example: 412VM58

Series	Style	Size	Color
412	V	M	58



20–30mmHg
Calf Sleeve Size Large in Black

412VL99



20–30mmHg
Calf Sleeve Size Large in Pink

412VL56

MERINO OUTDOOR SOCKS

for women & men

Compression Level

15–20mmHg

Also available in 20–30mmHg, see page 34

Features and Benefits

- » Soft and comfortable
- » Natural properties of wool make these socks odor resistant
- » Mesh flex zones for comfort and ventilation
- » Cushioned, padded soles protect against blisters
- » No compression in heel and instep for ease of donning
- » Thermoregulating for year-round comfort

Fiber Content

- » 67% Fine Merino Wool, 26% Nylon, 7% Spandex
- » Non-latex

Best Suited for

- » Tired, heavy, restless, aching legs
- » Outdoor sports and activities
- » Post-workout recovery



Sizing Chart

421	Women's Shoe	<7.5	7.5–9.5	10–12	12.5–15
	Men's Shoe	<6.5	6.5–8.5	9–11	11.5–14
	Ankle Circumference	7"–9.5" (18–24cm)	7.5"–10" (19–26cm)	8"–10.5" (20–27cm)	8.5"–11.5" (21–29cm)
	Size	S	M	L	X

Style and Colors

Closed Toe = ●

CHARCOAL 12	OLIVE 31
----------------	-------------

Calf (C)	●	●
----------	---	---

Product Code Example: 421CS12

Series	Style	Size	Color
421	C	S	12



15–20mmHg
Closed Toe Calf Size Large in Olive

421CL31



15–20mmHg
Closed Toe Calf Size Medium in Olive

421CM31

EVERSOFT DIABETIC SOCKS

for women & men

Compression Level

8–15mmHg

Also available in 18–25mmHg, see page 44

Features and Benefits

- » Thick, padded soles, heels, and toes for comfort and protection
- » Flat, low friction toe seam
- » Soft, stretchable knit-in band prevents pinching
- » Compression from ankle to knee, compression-free foot

Fiber Content

- » 67% drirelease (57% Polyester, 10% Cotton), 26% Nylon, 7% Spandex
- » Non-latex

Best Suited for

- » Individuals with diabetes
- » Individuals with limited dexterity or lower tolerance for compression
- » Individuals with sensitive skin



Sizing Chart					
160	Women's Shoe	<7.5	7.5–9.5	10–12	12.5–15
	Men's Shoe	<6.5	6.5–8.5	9–11	11.5–14
	Ankle Circumference	7"–9.5" (18–24cm)	7.5"–10" (19–26cm)	8"–10.5" (20–27cm)	8.5"–11.5" (21–29cm)
	Size	S	M	L	X

Style and Colors

Closed Toe = ●

WHITE

00

Calf (C)

●

Product Code Example: 160CS00

Series	Style	Size	Color
160	C	S	00



8–15mmHg
Closed Toe Calf Size Small in White

160CS00



8–15mmHg
Closed Toe Calf Size Large in White

160CL00

sigvaris



Medically Complex Edema

MEDICALLY COMPLEX EDEMA

The SIGVARIS GROUP line of products for medically complex edema (MCE) are for the treatment and management of conditions impacting the lymphatic and venous circulatory systems.

The product portfolio contains ready-to-wear garments for the active treatment and maintenance of lymphedema and for patients managing chronic venous diseases in the upper and lower extremities.

A sampling of these products is included in the following pages.

For more details and to request a complete MCE catalog, please ask your local SIGVARIS GROUP Territory Manager or email sigvariscares@sigvaris.com.

Medically Complex Edema Products

Product	Page
Compreflex Standard Calf & Foot	76
Compreflex Standard Calf	76
Compreflex Transition Calf	78
Compreflex Complete Calf	78
Coolflex Standard Calf	80
Assorted Liners	81
Assorted Foot Garments	82
Compreflex Strap Extender	83

Measuring Guide for Upper and Lower Inelastic Compression

Patient Name: _____ Phone #: _____ Date: _____
 Patient Address: _____ City: _____ State: _____ Zip: _____
 Prescribing Doctor's Name (Printed): _____
 NPI #: _____ Phone #: _____
 Doctor Signature: _____ ICD 10 Diagnosis Code: _____

Body Part*

☐ Calf ☐ Knee ☐ Foot


Inelastic Wraps

☐ Compreflex ☐ Coolflex ☐ Compreboot
☐ Light ☐ Standard ☐ Plus

Product style types may vary, please see the 2022 Sigvaris Catalog of Compression Solutions for Veno-Lymphatic Disorders for more information or call the US Customer Care Solution Center. Available Monday–Friday, 8am–7pm EST at **1.800.322.7744, ext. 1**.


Inelastic Wrap:

Lower Extremity Measurements

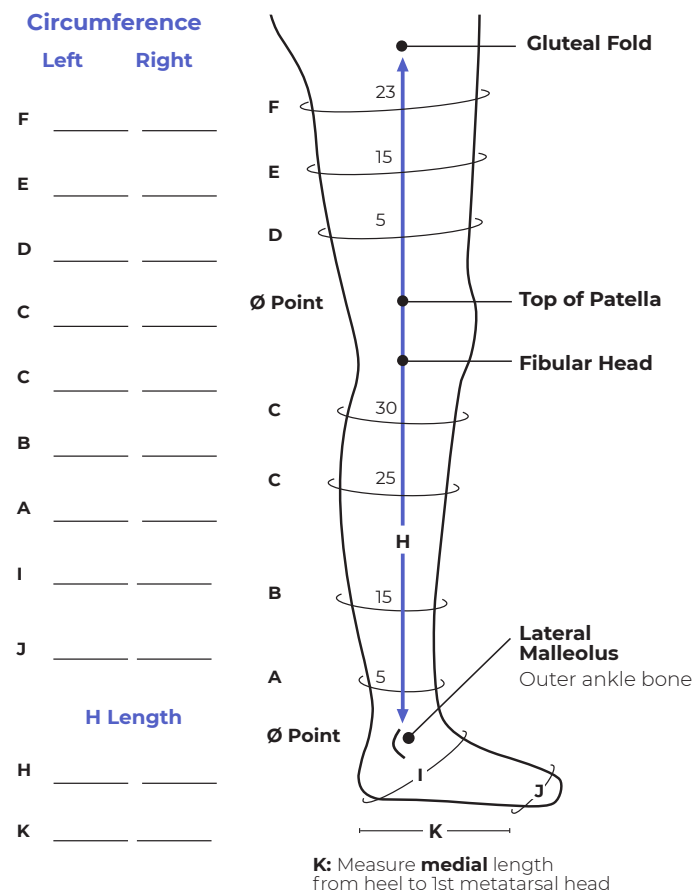

Inelastic Wrap:

Compreflex Calf Garment Measurements

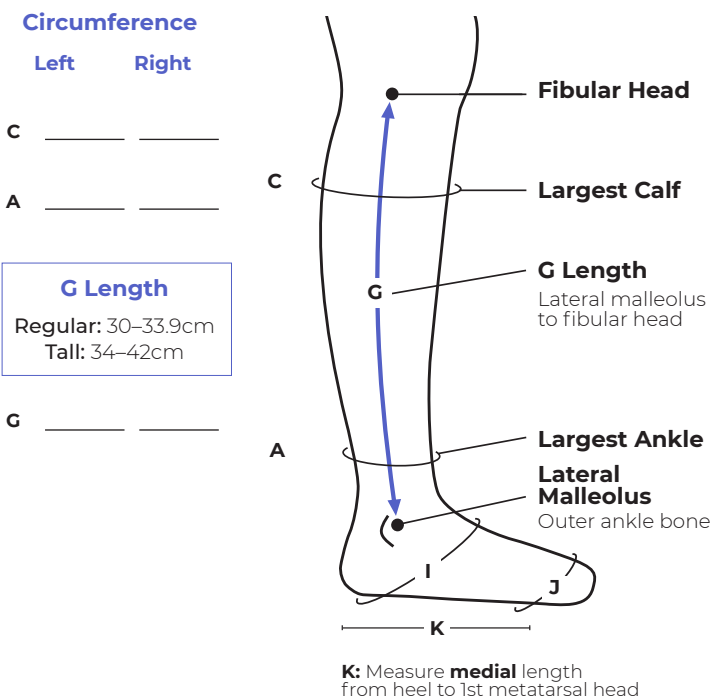

For a PDF of the product ordering form, scan the QR code or visit **sigvaris.group/mceforms**



Precise measurements are essential to guarantee the indicated level of compression and to ensure a proper fit. The measuring instructions included in the following pages are designed for your understanding.

The measurement chart and graphic for each product is also included on corresponding product specific order form that can be found online at: sigvaris.group/en-us/expertise/download-center

For additional guidance with measuring or fitting of a garment, please contact the Customer Care Solution Center, available Monday–Friday, 8am–7pm at **1.800.322.7744**, ext. 1.

CALF & FOOT

Circumferences:

For Compreflex Calf garments, measure circumferences where indicated on the measurement chart at the largest ankle and largest calf. Measurements may be recorded on the corresponding lines A and C.

For all other Calf garments, measure circumferences where indicated on the measurement chart, beginning with the dot 5cm above the **Lateral Malleolus** (outer ankle bone), continue to the 15cm, 25cm, and 30cm dots. Measurements may be recorded on the corresponding lines A, B, C, C (left/right).

Length:

Measure length from the **Lateral Malleolus** (outer ankle bone) to lateral aspect of the **Fibular Head**. Measurements may be recorded on the box G.

Foot:

Measure circumference of ankle bend and heel and record on line I. Measure circumference across metatarsal heads and record on line J. Measure length on medial side of foot from heel to 1st metatarsal head and record on line K.



Measurement tips

- 1 When working with a patient tasked with self-measuring, it is best to provide the measuring graphic (included in this guide) as a visual guide. Studies also show that the use of medical terminology may be confusing and even frightening for the patient. For the best result, limit the use of medical terminology.
- 2 Remind your patient, the tape or measurement tool should be held taut against the skin but should not dent the skin.
- 3 If patient has a caregiver or companion, it may be best to involve them for the placement of the tape or the reading of the measurements.
- 4 Using a body marking pen or makeup pencil may also help with recording landmarks on the limb for future reference.
- 5 Do not mark on fragile or broken skin.

Compreflex® Standard CALF & FOOT

20–50mmHg



Colors



Black



Beige

When to Use

- Adjustable ranges of resting compression is desired (20–50mmHg)
- Non-attached foot piece is desired
- No compression in the liner is desired
- Patient requires a calf and foot garment

Ideal for

- Chronic Venous Disease¹
- Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- Lymphedema - Active and Maintenance
- Non-specific or post-surgical edema



Compreboot® Standard

Variable compression
(see page 82 for details)



Basic Liner

Non-Compressive
(see page 81 for details)



Accutab® Devices

Accutab Devices
(visit website for details)



Compreflex Standard CALF

20–50mmHg



Colors



Black



Beige

When to Use

- Varying ranges of adjustable resting compression is desired (20–50mmHg)
- No compression in the liner is desired
- No Compreboot needed
- Patient requires compression in the calf only or a different foot option (size or boot)
- Ability to wear standard footwear over liner is desired

Ideal for

- Chronic Venous Disease¹
- Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- Lymphedema - Active and Maintenance
- Non-specific or post-surgical edema



Basic Liner

Non-Compressive
(see page 81 for details)



Accutab Devices

Accutab Devices
(visit website for details)






Variable compression Compreboot for added foot and ankle compression (see page 82 for details)

Compreflex Standard Calf & Foot Sizing Chart

		Small	Medium	Large	X Large	XX Large
	C	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
	A	19–26cm	21–30cm	26–36cm	31–41cm	36–46cm
Black	Regular	1401-BKR	1402-BKR	1403-BKR	1404-BKR	1405-BKR
	Tall	1401-BKT	1402-BKT	1403-BKT	1404-BKT	1405-BKT
Beige	Regular	1411-BKR	1412-BKR	1413-BKR	1414-BKR	1415-BKR
	Tall	1411-BKT	1412-BKT	1413-BKT	1414-BKT	1415-BKT

Tall garments ship with Regular length Compreboot. If Long Compreboot is required, purchase Compreflex Standard Calf and order desired Compreboot separately.

REQUIRED MEASUREMENTS

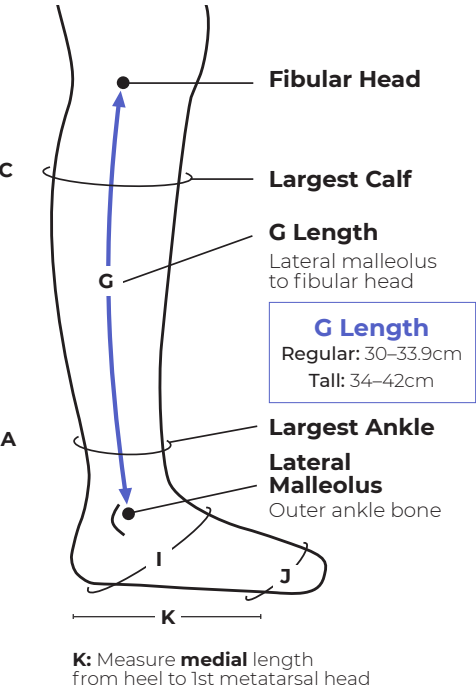

Compreflex Standard – Calf Sizing Chart (A–C)

		Small	Medium	Large	X Large	XX Large
	C	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
	A	19–26cm	21–30cm	26–36cm	31–41cm	36–46cm
Black	Regular	1401-NF-BKR	1402-NF-BKR	1403-NF-BKR	1404-NF-BKR	1405-NF-BKR
	Tall	1401-NF-BKT	1402-NF-BKT	1403-NF-BKT	1404-NF-BKT	1405-NF-BKT
Beige	Regular	1411-NF-BKR	1412-NF-BKR	1413-NF-BKR	1414-NF-BKR	1415-NF-BKR
	Tall	1411-NF-BKT	1412-NF-BKT	1413-NF-BKT	1414-NF-BKT	1415-NF-BKT



For a PDF of the product ordering form, scan the QR code or visit sigvaris.group/mceforms

Compreflex Transition

CALF

20-50mmHg



Colors



Black



Beige

When to Use

- Billable under Medicare code **6545-AW** while wound is active
- Patient requires a compressive liner with 10-15mmHg in the foot and ankle
- Ability to wear standard footwear over liner is desired

Ideal for

- Chronic Venous Disease¹
- Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- Lymphedema - Active and Maintenance
- Non-specific or post-surgical edema



Transition Liner

10-15 mmHg of mild compression in the foot and ankle
(see page 81 for details)



Accutab Devices

Accutab Devices
(visit website for details)

Compreflex Complete

CALF

20-50mmHg



Colors



Black



Beige

When to Use

- Patient requires a compressive liner with 10-15mmHg in the foot, ankle, and calf
- Ability to wear standard footwear over liner is desired

Ideal for

- Chronic Venous Disease¹
- Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- Lymphedema - Active and Maintenance
- Non-specific or post-surgical edema



Complete Liner


Mild compression in foot, ankle, and calf
(see page 81 for details)



Accutab Devices

Accutab Devices
(visit website for details)

REQUIRED MEASUREMENTS

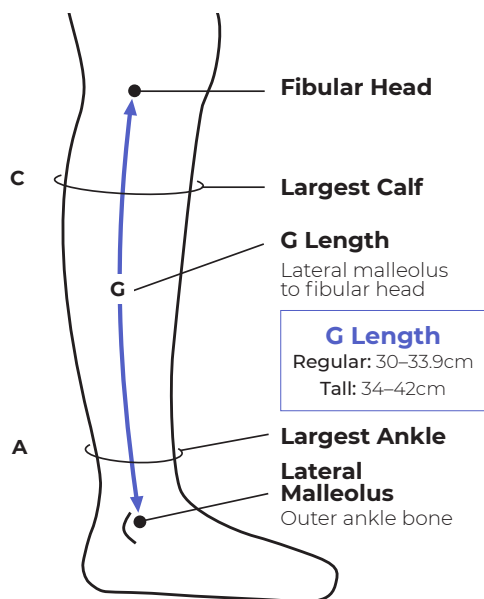

Compreflex Transition – Calf Sizing Chart (A-C)

		Small	Medium	Large	X Large	XX Large
	C	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
	A	19–26cm	21–30cm	26–36cm	31–41cm	36–46cm
Black	Regular	1401-UC-BKR	1402-UC-BKR	1403-UC-BKR	1404-UC-BKR	1405-UC-BKR
	Tall	1401-UC-BKT	1402-UC-BKT	1403-UC-BKT	1404-UC-BKT	1405-UC-BKT
Beige	Regular	1411-UC-BKR	1412-UC-BKR	1413-UC-BKR	1414-UC-BKR	1415-UC-BKR
	Tall	1411-UC-BKT	1412-UC-BKT	1413-UC-BKT	1414-UC-BKT	1415-UC-BKT



Compreflex Complete Calf

20-50mmHg

Medium Regular in Beige

1412-FC-BKR

Compreflex Complete – Calf Sizing Chart (A-C)

		Small	Medium	Large	X Large	XX Large
	C	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
	A	19–26cm	21–30cm	26–36cm	31–41cm	36–46cm
Black	Regular	1401-FC-BKR	1402-FC-BKR	1403-FC-BKR	1404-FC-BKR	1405-FC-BKR
	Tall	1401-FC-BKT	1402-FC-BKT	1403-FC-BKT	1404-FC-BKT	1405-FC-BKT
Beige	Regular	1411-FC-BKR	1412-FC-BKR	1413-FC-BKR	1414-FC-BKR	1415-FC-BKR
	Tall	1411-FC-BKT	1412-FC-BKT	1413-FC-BKT	1414-FC-BKT	1415-FC-BKT



For a PDF of the product ordering form, scan the QR code or visit sigvaris.group/mceforms

Coolflex™ Standard CALF

20–50mmHg



Color



Black

When to Use

- Varying ranges of adjustable resting compression is desired (20–50mmHg)
- No compression in the liner is desired
- Simplified one-handed donning is desired
- More breathable fabric is desired
- Desire to select separate foot piece

Ideal for

- Chronic Venous Disease¹
- Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis
- Fragile, sensitive skin
- Well-managed Lymphedema
- Lipedema

One-Handed Donning

Spacer Fabric is highly breathable for cool-wearing comfort


Durable, metal hook closures for maximum strength and durability

Basic Liner

Non-Compressive
(see page 81 for details)

Coolflex Standard Calf Sizing Chart (A–C)

SIZE	Small	Small Max	Medium/Large	Medium / Large Max	X Large / XX Large
C	28–39cm	35–50cm	34–50cm	45–64cm	45–64cm
C	28–39cm	35–50cm	34–50cm	45–64cm	45–64cm
B	23–33cm	27–38cm	29–43cm	35–49cm	37–54cm
A	16–26cm	16–26cm	23–33cm	23–33cm	30–42cm
Left					
Regular	2101-NF-BKR-L	2101-NF-BMR-L	2103-NF-BKR-L	2103-NF-BMR-L	2105-NF-BKR-L
Tall	2101-NF-BKT-L	2101-NF-BMT-L	2103-NF-BKT-L	2103-NF-BMT-L	2105-NF-BKT-L
Right					
Regular	2101-NF-BKR-R	2101-NF-BMR-R	2103-NF-BKR-R	2103-NF-BMR-R	2105-NF-BKR-R
Tall	2101-NF-BKT-R	2101-NF-BMT-R	2103-NF-BKT-R	2103-NF-BMT-R	2105-NF-BKT-R

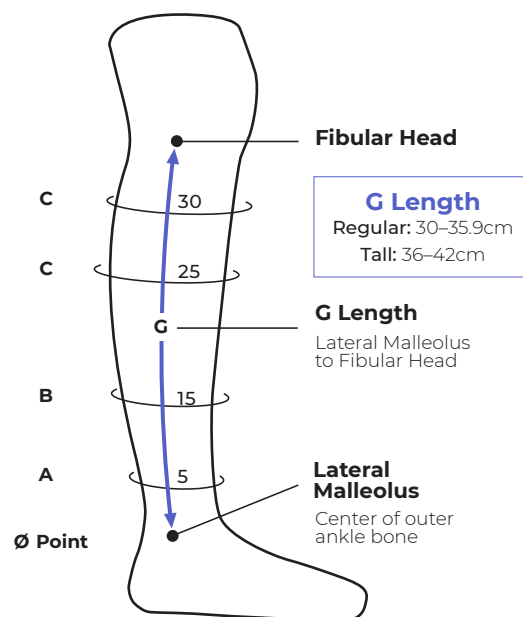

Basic Liner

Non-compressive

Features

- Non-compressive liner for the protection of the skin
- Antimicrobial wash for bacteriostatic properties and odor control
- Flat seams for added comfort
- Available in Heathered Silver color only

Size: One size fits all	
Pair	10 Pair Pack
1700 - NK1	1700 - NKC
Included With Compreflex Standard Calf & Foot Compreflex Standard Calf	



Transition Liner

10-15mmHg

Features

- Mild compression in foot and ankle only
- Antimicrobial wash for bacteriostatic properties and odor control
- Extra thick sole from toe to heel for added comfort
- Available in Black and Beige

Transition Liner Sizing Chart (Included with Compreflex Transition)						
SIZE	Ankle Circ. (A)	Calf Circ. (C)	Women's Shoe	Men's Shoe	Pair (Black)	Pair (Beige)
Small	19-26cm	29-39cm	<7½	<6½	1601-CK	1611-CK
Medium	21-30cm	34-44cm	7.5-9.5	6.5-8.5	1602-CK	1612-CK
Large	26-36cm	40-50cm	10-12	9-11	1603-CK	1613-CK
X Large	31-41cm	46-56cm	12.5-15	11.5-14	1604-CK	1614-CK
XX Large	36-46cm	54-64cm	12.5-15	11.5-14	1605-CK	1615-CK



Complete Liner

10-15mmHg

Features

- Mild compression in foot and ankle with graduated compression in calf
- Antimicrobial wash for bacteriostatic properties and odor control
- Extra thick sole from toe to heel for added comfort
- Available in Black and Beige

Complete Liner Sizing Chart (Included with Compreflex Complete)						
SIZE	Ankle Circ. (A)	Calf Circ. (C)	Women's Shoe	Men's Shoe	Pair (Black)	Pair (Beige)
Small	19-26cm	29-39cm	<7½	<6½	1601-CK-F1	1611-CK-F1
Medium	21-30cm	34-44cm	7.5-9.5	6.5-8.5	1602-CK-F1	1612-CK-F1
Large	26-36cm	40-50cm	10-12	9-11	1603-CK-F1	1613-CK-F1
X Large	31-41cm	46-56cm	12.5-15	11.5-14	1604-CK-F1	1614-CK-F1
XX Large	36-46cm	54-64cm	12.5-15	11.5-14	1605-CK-F1	1615-CK-F1



Compreboot Lite Foot

Variable Compression



Features

- Ultra-thin profile that easily fits into most footwear
- Highly adjustable design may be trimmed to fit
- Provides mild, therapeutic compression

Color



Black



Compreboot Standard Foot

Variable Compression



Features

- Four-way stretch for firm compression around ankle
- Ideal for mild to moderate chronic edema
- Low stretch material conforms to foot shape

Colors



Black



Beige



Compreboot Plus Foot

Variable Compression



Features

- Chip foam lining creates high / low pressure ridges
- Removable gripper patches provide traction
- Provides moderate, therapeutic compression

Color



Black



Coolflex Standard Foot

Variable Compression



Features

- Low profile design allows boot to fit in most footwear*
- Spacer Fabric is highly breathable for cool-wearing comfort
- Durable metal clasps for closure

Color



Black



* Data on file

Foot compression garments are pictured with the Basic Liner. Basic Liner is not included.

Circumference


Left Right

I _____

J _____

Length

K _____

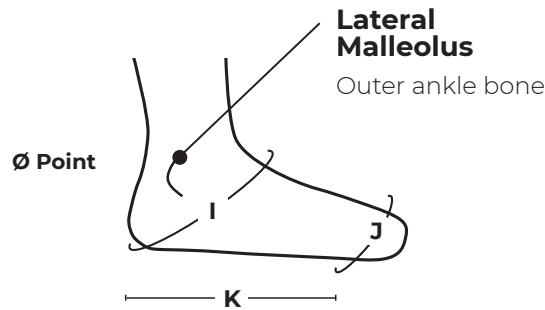

K: Measure **medial** length from heel to 1st metatarsal head

Measuring Instructions

Foot Size:

- I:** Measure circumference of ankle bend and heel
- J:** Measure circumference across metatarsal heads
- K:** Measure length from heel to 1st metatarsal head

Compreboot Lite Foot Size Chart (I-K)
(Available in Black Only)

	Small / Medium / Large		X Large / XX Large	
	Regular	Long	Regular	Long
I	28–36cm	28–36cm	≤44cm	≤44cm
J	22–26cm	22–26cm	≤33cm	≤33cm
K	14–18cm	19–23cm	20–22cm	25–27cm
Black	1803-BLR	1803-BLL	1805-BLR	1805-BLL

Compreboot Standard Foot Size Chart (I-K)

	Small		Medium / Large		X Large / XX Large	
	Regular	Long	Regular	Long	Regular	Long
I	28–36cm	28–36cm	≤39cm	≤39cm	≤44cm	≤44cm
J	22–26cm	22–26cm	≤30cm	≤30cm	≤33cm	≤33cm
K	14–18cm	19–23cm	18–20cm	23–25cm	20–22cm	25–27cm
Black	1801-BTR	1801-BTL	1803-BTR	1803-BTL	1805-BTR	1805-BTL
Beige	1811-BTR	1811-BTL	1813-BTR	1813-BTL	1815-BTR	1815-BTL

Compreboot Plus Foot Size Chart (I-K)
(Available in Black Only)

	Small		Medium / Large		X Large / XX Large	
	Regular	Long	Regular	Long	Regular	Long
I	28–36cm	28–34cm	≤39cm	≤39cm	≤44cm	≤44cm
J	22–26cm	22–25cm	≤30cm	≤28cm	≤33cm	≤33cm
K	14–18cm	19–23cm	18–20cm	23–25cm	20–22cm	25–27cm
Black	1801-BPR	1801-BPL	1803-BPR	1803-BPL	1805-BPR	1805-BPL

Coolflex Standard Foot Size Chart (I-K)
(Available in Black Only)

	Small		Medium / Large		X Large / XX Large	
	Regular	Long	Regular	Long	Regular	Long
I	28–38cm	28–38cm	30–43cm	30–43cm	34–49cm	34–49cm
J	22–26cm	22–26cm	25–30cm	25–30cm	28–33cm	28–33cm
K	14–18cm	19–21cm	18–20cm	21–23cm	20–22cm	23–25cm
Left						
Black	2101-BTR-L	2101-BTL-L	2103-BTR-L	2103-BTL-L	2105-BTR-L	2105-BTL-L
Right						
Black	2101-BTR-R	2101-BTL-R	2103-BTR-R	2103-BTL-R	2105-BTR-R	2105-BTL-R

Compreflex Strap Extender

- Provides up to 10cm of additional circumferential size range to Compreflex calf wraps
- Detaches easily
- Potential inexpensive solution for patients outside of standard size ranges of Compreflex calf wraps

	1 Unit	5 Unit Pack	10 Unit Pack
Black	1401-SE1	1401-SE5	1401-SEB
Beige	1411-SE1	1411-SE5	1411-SEB



For a PDF of the product ordering form, scan the QR code or visit sigvaris.group/mceforms



The best choice for
**conventional,
precise-sized,
comfortable,
and affordable
compression**

DYNAVEN

Conventional Compression

Why choose **DYNAVEN**?

- Affordable compression without compromising quality
- Support for legs following a vein procedure
- Wide range of compression levels and sizes

DYNAVEN Products		
Product	Series Number	Page
OPAQUE FOR WOMEN & MEN	970	86
OPAQUE FOR MEN	920	86
SHEER FOR WOMEN	980	88
CUSHIONED FOR WOMEN & MEN	960	88

970 opaque WOMEN & MEN

Dynaven socks and hosiery provide opaque coverage and support for legs following a vein procedure or for everyday wear.

Features

- Glide on & off easily
- Reinforced heel area
- Roomy toe area
- Wide top bands

Compression

15–20mmHg
20–30mmHg
30–40mmHg



Silicone grip-top band



Light Beige



Black

920 opaque MEN

Dynaven opaque ribbed socks provide support for legs following a vein procedure or for everyday wear.

Features


- Ribbed sock style
- Glide on & off easily
- Roomy toe area
- Wide top bands

Compression


15–20mmHg
20–30mmHg
30–40mmHg



Silicone grip-top band

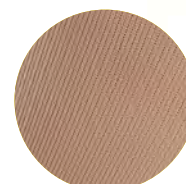

Light Beige

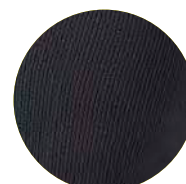

Black

Sizing

Ankle circumference	small 7"–8.5" (18–21.5cm)		medium 8.5"–10" (21.5–25.5cm)		large 10"–11.5" (25.5–29cm)		x-large 11.5"–13" (29–33cm)	
Calf circumference	small 11"–15.5" (28–39cm)		medium 12.5"–17.5" (31.5–45cm)		large 14"–20" (35.5–51cm)		x-large 16"–23" (40.5–58cm)	
Calf length	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)
Thigh circumference	small 18"–26" (45.5–66cm)		medium 19"–28.5" (48–73cm)		large 20.5"–33" (52–84cm)		x-large 24"–36" (61–92cm)	
Leg length	short ≤28.5" (72cm)	long >28.5" (72cm)	short ≤28.5" (72cm)	long >28.5" (72cm)	short ≤28.5" (72cm)	long >28.5" (72cm)	short ≤28.5" (72cm)	long >28.5" (72cm)
Hip circumference guidelines	<45.5" (115cm)		<51" (129.5cm)		<60.5" (154cm)		N/A	
Size	SS	SL	MS	ML	LS	LL	XS*	XL*

Women's shoe sizes **Men's shoe sizes**
Short 5.5–9.5 Long 8–12 Short 7–11 Long 9.5–14

* XS & XL not available in Dynaven pantyhose styles

Fiber contents

75% Nylon, 25% Spandex
(Content can vary by style)
Not made with natural rubber latex

Styles and colors

Closed toe = ● Open toe = ○	Light beige (66)	Black (99)
970 Women's opaque calf, calf w/grip-top, thigh w/grip-top, and pantyhose (C, C/S, N, & P)	●	●
970 Men's opaque thigh w/grip top (N)	●	●
970 Unisex opaque calf, calf w/grip-top, thigh w/grip-top, and pantyhose (CO, CO/S, NO, & PO)**	○	○

** Dynaven unisex opaque pantyhose only available in 20–30mmHg and 30–40mmHg

Product code example: 972CSLW66

Dynaven opaque 20–30mmHg closed-toe calf, small long for women, in light beige.

Series	mmHg level	Style	Size	Gender/ open toe	Color
97	2	C	SL	W	66

Sizing

Ankle circumference	small 7"–8.5" (18–21.5cm)		medium 8.5"–10" (21.5–25.5cm)		large 10"–11.5" (25.5–29cm)		x-large 11.5"–13" (29–33cm)	
Calf circumference	small 11"–15.5" (28–39cm)		medium 12.5"–17.5" (31.5–45cm)		large 14"–20" (35.5–51cm)		x-large 16"–23" (40.5–58cm)	
Calf length	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)
Size	SS	SL	MS	ML	LS	LL	XS	XL

Men's shoe sizes
Short 7–11 Long 9.5–14

Styles and colors

Closed toe = ● Open toe = ○	Light beige (66)	Black (99)
920 Men's opaque ribbed calf and calf w/grip-top (C & C/S)	●	●

Product code example: 922CMLM99

Dynaven opaque 20–30mmHg closed-toe calf, medium long for men, in black.

Series	mmHg level	Style	Size	Gender	Color
92	2	C	ML	M	99

Fiber contents

80% Nylon, 20% Spandex
Not made with natural rubber latex

980 sheer WOMEN

Dynaven sheer hosiery offers medical grade compression and features sheer fabric for a fashionable look. Ideal for everyday wear, Dynaven sheer hosiery supports leg and vein health.

Features

- Sheer fabric for a fashionable look
- Glide on & off easily
- Lace silicone grip-top band for thigh w/grip-top styles
- Cotton-lined gusset for pantyhose styles

Compression

15–20mmHg
20–30mmHg



Silicone grip-top band



Beige



Black

960 cushioned WOMEN & MEN


Dynaven Cushioned socks feature cushioned heel, sole, and toe areas. Ideal for everyday wear or following a vein procedure, the Dynaven Cushioned sock gives legs the support and comfort they need to go the distance.

Features


- Glide on & off easily
- Cushioned heel, sole, & toe areas
- Wide top bands
- Single sizing system for both men and women

Compression

15–20mmHg
20–30mmHg

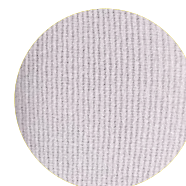

White

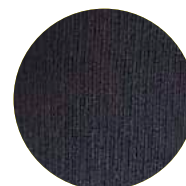

Black

Sizing

Ankle circumference	small 7"–8.5" (18–21.5cm)		medium 8.5"–10" (21.5–25.5cm)		large 10"–11.5" (25.5–29cm)	
Calf circumference	small 11"–15.5" (28–39cm)		medium 12.5"–17.5" (31.5–45cm)		large 14"–20" (35.5–51cm)	
Calf length	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)
Thigh circumference	small 18"–26" (45.5–66cm)		medium 19"–28.5" (48–73cm)		large 20.5"–33" (52–84cm)	
Leg length	short ≤28.5" (72cm)	long >28.5" (72cm)	short ≤28.5" (72cm)	long >28.5" (72cm)	short ≤28.5" (72cm)	long >28.5" (72cm)
Hip circumference guidelines	<45.5" (115cm)		<51" (129.5cm)		<60.5" (154cm)	
Size	SS	SL	MS	ML	LS	LL

Women's shoe sizes

Short 5.5–9.5 Long 8–12

Fiber contents

65% Nylon, 35% Spandex

(Content can vary by style)

Not made with natural rubber latex

Styles and colors

Closed toe = ● Open toe = ○	Beige (77)	Black (99)
980 Women's Sheer calf, thigh-high w/grip-top, and pantyhose (C, N, & P)*	● * ○	● * ○

* Dynaven sheer pantyhose only available in closed toe

Product code example: 982NSSW77

Dynaven sheer 20–30mmHg closed-toe thigh, small short for women, in beige.

Series	mmHg level	Style	Size	Gender/ open toe	Color
98	2	N	SS	W	77

Sizing

Ankle circumference	small 7"–8.5" (18–21.5cm)		medium 8.5"–10" (21.5–25.5cm)		large 10"–11.5" (25.5–29cm)		x-large 11.5"–13" (29–33cm)	
Calf circumference	small 11"–15.5" (28–39cm)		medium 12.5"–17.5" (31.5–45cm)		large 14"–20" (35.5–51cm)		x-large 16"–23" (40.5–58cm)	
Calf length	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)	short ≤16" (41cm)	long >16" (41cm)
Size	SS	SL	MS	ML	LS	LL	XS	XL

Women's shoe sizes

Short 5.5–9.5 Long 8–12

Men's shoe sizes

Short 7–11 Long 9.5–14

Styles and colors

Closed toe = ●	White (00)	Black (99)
960 Unisex cushioned calf (C)	●	●

Product code example: 961CLS00

Dynaven cushioned 15–20mmHg closed-toe calf, large short, in white.

Series	mmHg level	Style	Size	Color
96	1	C	LS	00

Fiber contents

82% Nylon, 18% Spandex

Not made with natural rubber latex



ACCESSORIES

Help Yourself

Why choose **Accessories**?

- Aid in putting on and removing stockings
- Stocking cleaning to maximize usage
- Adhesives and extra liners for added comfort

Accessories

Product	Page
Gloves	92
Sim-Slide	92
Magnide On/Off	92
Cotton Underliner	93
Washing Solution	93
IT STAYS!®	93
Foot Slip	94
Sigvaris DOFF N' DONNER	94
Sigvaris DOFF N' DONNER CONE	94
MELANY®	94

Gloves

Rubber Gloves – Ridged

The practical and protective way to apply and remove your compression stockings.



Package: 12 per Pack

Size	Product Code
Small	591R400S
Medium	591R400M
Large	591R400L
X-Large	591R400X

Package: Single Pair

Size	Product Code
Small	591RPRS
Medium	591RPRM
Large	591RPRL
X-Large	591RPRX

Glove Sizing Chart

Measure Circumference of the Hand at the Palm

6.5"–7" (16.5–18cm)	7.5"–8" (19–20.5cm)	8.5"–9" (21.5–23cm)	9.5"–10" (24–25.5cm)
Small	Medium	Large	X-Large



For more Sigvaris videos, including donning tips and product demonstrations, visit our YouTube channel: youtube.com/sigvarisgroupusa

Caution: Rubber Gloves contains natural rubber latex which may cause allergic reaction

Latex-Free Gloves

Gripping donning gloves recommended for use in a non-latex environment.



Package: 12 per Pack

Size	Product Code
Small	592R400S
Medium	592R400M
Large	592R400L
X-Large	592R400X

Package: Single Pair

Size	Product Code
Small	592RPRS
Medium	592RPRM
Large	592RPRL
X-Large	592RPRX

Donning Aids



Sim-Slide®



Video: How to Use the Sim-Slide
To watch the video go to: bit.ly/SIGssyt

Sim-Slide

Size	Product Code
Small	587SLIDESM
Medium	587SLIDEMD
Large	587SLIDELG
X-Large	587SLIDEXL

Sizing Chart

For use with Open toe styles

Women's Shoe Size	≤4–6	6.5–8	8.5–9.5	≥10
Men's Shoe Size	≤3–4.5	5–6.5	7–8	≥8.5
Size	S	M	L	XL



Magnide On/Off®



Video: How to Use the Magnide On/Off
To watch the video go to: bit.ly/SIGmofyt

Magnide On/Off

Size	Product Code
Medium	587MAGMD
Large	587MAGLG
X-Large	587MAGXL

Sizing Chart

For use with Closed toe styles

Women's Shoe Size	≤8.5	9–12	≥12.5
Men's Shoe Size	≤7.5	8–11	≥11.5
Size	M	L	XL*

*XL suitable for extra-wide feet

Donning Aid



Cotton Underliner

10–15mmHg stocking. Can be used as an underlayer to protect fragile skin, secure wound dressings, add 10–15mmHg of compression, or create an easy glide surface to aid in putting on and taking off compression stockings. For sizing, please refer to the Natural Rubber Calf sizing on page 39.

Package: 2 per pack

Size	Product Code
Small / Short	588USS
Small / Long	588USL
Medium / Short	588UMS
Medium / Long	588UML
Large-XL / Short	588ULS
Large-XL / Long	588ULL

Washing Solution



Washing Solution

Gentle washing solution specifically formulated to help maintain the quality of SIGVARIS GROUP compression socks and stockings. Also great for washing delicate garments!

Package Options

Contains	Product Code
Six 16 oz. Bottles (473ml)	586W900
One 16 oz. Bottle (473ml)	586W950
Twelve 2 oz. Bottles (60ml)	586W975
One 2 oz. Bottle (60ml)	586W985

Body Adhesive



IT STAYS!

A gentle roll-on body adhesive that washes off with water and may be used to hold compression garments in place against the skin. IT STAYS! is pliable and moves with the body.

Package Options

Contains	Product Code
Twelve 2 oz. Bottles (60ml)	581I300
One 2 oz. Bottle (60ml)	581I350



Donning Aids



Foot Slip

Comfortable applicator for use with open toe stockings.

Package: 6 per pack

Size	Product Code
One Size	580F200



Sigvaris DOFF N' DONNER

A soft device that allows you to quickly guide a compression sock or stocking up and down your leg or arm, making it simple to put on and remove all styles of graduated compression. It is perfect for both self-donning and caregiver use. Non-latex.

Size	Product Code
One Size	583D100



Sigvaris DOFF N' DONNER CONE

This loading device makes it easy to roll stockings onto the Sigvaris DOFF N' DONNER. The bottom of the base firmly attaches to most clean, flat surfaces via suction.



Video: How to Use the Sigvaris DOFF N' DONNER
To watch the video go to:
bit.ly/sigdnd

Size	Product Code
One Size	583D200

Caution: this product contains natural rubber latex which may cause allergic reactions



MELANY

The donning and doffing device for compression stockings in case of limited mobility.

Sizing Chart				
Size	X-SMALL	SMALL	MEDIUM	LARGE
Calf Circumference	10"-12" (25-30cm)	12"-15" (30-38cm)	15"-18" (38-46cm)	18"-22" (46-56cm)
Product Code	579MXS	579MSM	579MMD	579MLG

Measure at the widest part of the calf



Video: How to Use the MELANY
To watch the video go to:
goto.sigvaris.com/bbiMdz

WARNING: This product can expose you to chemicals including Carbon black, which are known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov.

Symptoms / Indications	Compression Level at the Ankle
<ul style="list-style-type: none"> Tired, achy legs Occupational or evening edema Leg discomfort from long hours of standing or sitting Prevent varicosities & swelling during pregnancy Preventative for legs predisposed to risk Reduce swelling during travel Prevent DVT during travel 	15–20mmHg
<ul style="list-style-type: none"> Patient with diabetes who experiences edema (see Cautions below) 	18–25mmHg
<ul style="list-style-type: none"> Heavy, fatigued, aching legs Mild edema in lower extremities Mild varicosities with minimal edema Mild varicosities during pregnancy Lymphedema upper extremities Post-procedure of small veins Prevention or management of DVT 	20–30mmHg
<ul style="list-style-type: none"> Moderate varicosities with mild to moderate edema (also during pregnancy) Post-procedure of larger veins to maintain treatment success CEAP C3 (edema), C4 (skin changes without ulceration), C5 (skin changes with healed ulceration), C6 (skin changes with active and / or recurrent ulcerations) Management and treatment of DVT or Post-Thrombotic Syndrome Superficial phlebitis Lymphedema after decongestant therapy to maintain reduction Orthostatic / postural hypotension Post-fracture, post-traumatic edema <p><i>Note: A prescription is recommended for stockings with the indications of 30mmHg or higher</i></p>	30–40mmHg
<ul style="list-style-type: none"> Lymphedema after decongestant therapy to maintain reduction Post-Thrombotic Syndrome Varicosities and / or edema 	40mmHg or higher
<ul style="list-style-type: none"> Severe arterial insufficiency, intermittent claudication, ischemia Uncontrolled congestive heart failure 	Contraindications
<ul style="list-style-type: none"> Skin sensitivities or allergies Advanced neuropathy with sensory loss in leg or foot Diabetes with arterial compromise Confinement to bed or non-ambulatory use unless otherwise prescribed by the physician Eczema, hypodermatitis, dermatosclerosis, dermatitis 	Cautions
<p><i>No liability accepted for non-observance of contraindications and cautions.</i></p> <p>Fronek HS, Bergan JJ, et al. The Fundamentals of Phlebology: Venous Disease for Clinicians. 2004. pg 151. Blaettner, et al. Leg Symptoms of Healthy People and Their Treatment with Compression Hosiery. Phlebology 2008. Compression therapy of the extremities. Partsch, Rabe, Stemmer. ISBN: 2.85480.770.7.</p>	

Term	Definition	Sigvaris Usage
Chronic Venous Disease (CVD)	A disease state comprised of a wide range of disorders including tired, achy legs, varicose veins, Chronic Venous Insufficiency (CVI), skin issues, and venous leg ulcers (open and healed).	SIGVARIS GROUP manufactures Medical and Well Being products to help reduce the risk of, improve symptoms of, and provide treatment for CVD.
Double-Covered Yarn	Spandex fibers are covered twice with yarn to prevent bare spandex from touching the skin for improved wearing comfort.	Used in nearly all SIGVARIS GROUP graduated compression socks and hosiery to create products that are easy to put on and take off, while also providing greater comfort.
Deep Vein Thrombosis (DVT)	A blood clot in a deep vein, most commonly in the lower leg or thigh.	SIGVARIS GROUP cares about DVT prevention. Clinical evidence supports the use of graduated compression to help prevent DVTs and the more serious complication of PE (Pulmonary Embolism), which can be life-threatening.
Graduated Compression v. Anti-Embolism	Graduated compression is for ambulatory patients, whereas anti-embolism stockings are for bedridden patients.	SIGVARIS GROUP offers true graduated compression, meaning products are tightest at the ankle and decrease in pressure going up the leg to help improve circulation.
Hypoallergenic	Designed to minimize the chance of an allergic reaction.	SIGVARIS GROUP offers non-latex products and cotton fabrics for those with sensitive skin and allergies. Also, many SIGVARIS GROUP accessories are designed for consumers with sensitive skin.
ISO Certified	The International Organization for Standardization, a worldwide federation of national standards bodies from over 130 countries. The United States member body is identified as ANSI (American National Standards Institute).	SIGVARIS GROUP follows ISO Standard 13485:2016 for our graduated compression products manufactured in North and South America. For our MCE products manufactured in Holland, Michigan, we follow ISO 9001.
Microfiber	An extremely fine synthetic fiber knitted into textiles with the texture and drape of natural-fiber fabric. This fabric has excellent breathability.	Used to create products with a soft and comfortable feel. Find this fiber in 840 Soft Opaque, 830, 143 & 183 Microfiber Patterns, and 820 Microfiber.
mmHg (Millimeters of Mercury)	Defined as the extra pressure generated by a column of mercury one millimeter high; the same unit of measurement used for blood pressure.	SIGVARIS GROUP products are offered in several ranges of compression levels based on this measurement.
Merino Wool	Merino is the softest and finest wool available in the world and is inherently itch-free.	SIGVARIS GROUP uses this premium wool yarn to create soft products that are thermoregulating for use in all seasons. Used in 422 Thermoregulating Wool, 421 MERINO OUTDOOR SOCKS, and 152 & 192 ALL-SEASON MERINO WOOL.
Sea Island Cotton	Sea Island Cotton is the rarest cotton in the world and represents the finest in cotton softness.	SIGVARIS GROUP uses the highest-quality yarns in the industry to create products that are best in class. Look for this fine quality cotton in our 220 Sea Island and 151 & 191 SEA ISLAND COTTON SOCKS.
Sensinnov	A patented SIGVARIS silicone top band that gently grips the skin and keeps the band in place all day.	This top band is featured in 780 Sheer, 840 Soft Opaque, 710 Patterns, and 820 Microfiber to help keep thigh stockings in place all day long.
Supima Cotton SUPIMA® <small>WORLD'S FINEST COTTONS</small>	Supima accounts for only about three percent of annual cotton production in the United States. Its fineness and longer staple length make it a premium cotton fiber.	SIGVARIS GROUP uses this premium cotton yarn in the 230 Cotton, 220 Sea Island and 151 & 191 SEA ISLAND COTTON SOCKS, 146 & 186 CASUAL COTTON, and 360, 142 & 182 Cushioned Cotton products, making them comfortable and ideal for those with sensitive skin.

SIGVARIS GROUP Dealer Portal

The SIGVARIS GROUP Dealer Portal is your online location for securely ordering SIGVARIS GROUP products and managing your account. Place and track your orders, review order history, chat with Customer Care, and much more at **sigvaris-online.com**.

CAN'T FIND THE PRODUCT YOU ARE SEARCHING FOR?

Type in the product SKU in the search function and select for ordering.

DEALER PORTAL BENEFITS

24/7 access-order any time!

Sizing Wizard

Three ways to securely place an order online:
Catalog, Quick Entry or Excel Upload

Access to order history and tracking

Shipping options including drop ship and easy access to an address table for quick shipment of reorders

Check your account balance and print invoices

New product launch information

Current promotions

Inelastic compression order forms

Email **sigvariscares@sigvaris.com** and you will receive your active account within one business day!



SIGVARIS GROUP Education & Training



SIGVARIS GROUP has several new and exciting educational opportunities for 2022 to help you get a leg up on leg health knowledge. These opportunities include online learning, webinars, and custom training sessions to educate fitters, clinicians, and therapists. Additionally, we offer a variety of courses with opportunities to earn continuing education credits.

Learn more at **sigvariseducation.com**.

Become a SIGVARIS Certified Fitter!

Contact your Territory Manager
or email **medical.education@sigvaris.com**.

SIGVARIS
GROUP

4 Fronek HS, Bergan JJ, et al. The Fundamentals of Phlebology: Venous Disease for Clinicians. 2004. pg 151. Blaettner, et al.

Sigvaris, SIGVARIS GROUP, DYNAVEN, MELANY, Sensinnov, Compreflex, Compreboot, and Accutab are registered trademarks of SIGVARIS AG, CH-9014 St.Gallen/Switzerland, in many countries worldwide.
© Copyright 2022 SIGVARIS, Inc.

Coolflex is a trademark of SIGVARIS AG, CH-9014, St. Gallen/Switzerland, in many countries worldwide.

Lycra is a registered trademark of INVISTA.

Supima is a registered trademark of the Supima Association of America Corporation.

NILIT is a registered trademark of Nilit. All rights reserved.

drirelease is copyright ©2022 *drirelease*. *drirelease* is an Optimer Brands company. All rights reserved.

Sim-Slide and Magnide are registered trademarks of Arion Holding B.V.

® IT STAYS is a registered trademark of New Horizons Diversified Interests.

SIGVARIS GROUP

ORDERING

Online: sigvaris-online.com

Call: 1-800-322-7744, ext. 1

Fax: 1-800-481-5488

CUSTOMER CARE

Monday–Friday, 8am–7pm (EST)

QUALITY STANDARD

Certified Quality System ISO 13485.

sigvaris.com



SigvarisUSA

